



thermaeye[®]
PLUS

Thinking about your patients

Thinking about your patients

Recent conclusion by the prestigious MGD workshop report that MGD is likely the leading cause of all dry eye is supported by a large body of evidence-based medicine reporting the compromise to meibomian gland function negatively impacts all aspects of ocular surface health.

Intense Pulsed Light therapy appear as a safe and efficient threatment in relieving symptoms and signs of DED-MGD eyes. No significant complications has been observed.

Benefits

- V** Vascular
- A** Anti – inflammatory
- A** Anti – Microbial
- T** Tissue regeneration

Pathologies

- Rosacea
- Telangiectasia
- MGD
- Brown spots
- Wrinkles




Expanding your Practice

Different Filters Multiple



Courtesy of: P. Campolmi, MD - G. Cannarozzo, MD - P. Bonan, MD.
Florence, Italy. (Photos using polarized light)

520nm. It is the perfect filter to perform treatments of vascular lesions and skin pigmentation to patients with grade III of skin type.

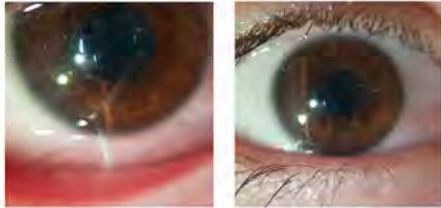

Courtesy of Dr. Carlos Verges

650 nm. The right combination makes this filter ideal for treating MGD in all skin types, up to grade VI phototype.



Courtesy of: A. Pulvirenti, MD - Catania, Italy.

SA - 800nm. Thanks to the ultraviolet emission of this filter, we can shoot directly on the eyelid to treat the Demodex in a more efficient way. In addition, thanks to this filter we stimulate the production of new collagen, improving the tone, texture and luminosity of the skin, giving it a lifting effect without the need for

Care your patients

IPL helps to improve the results in each treatment



Dry eye
Refractive
Glaucoma
Oculoplasty

Results

Many studies confirm that IPL achieves statistically significant improvements in the relief of clinical signs and symptoms of patients affected by MGD related to DED, even after few treatments.


The high versatility of ThermaeyePlus device allows doctors to set customized treatments according to patient's characteristics in order to guarantee maximum efficacy and tolerability.

Advanced IPL Technology

Pulse Duration - TRT - Pulse Duration

Fractionated pulse: security guarantee


The pulse of ThermaeyePlus can be divided up to three pulses. The fractionated pulses allow to use higher fluences while protecting the skin. The Doctor can select the duration and energy of each individual pulse, and also set the interval. This intelligent technology protects the skin and prevents thermal damage.

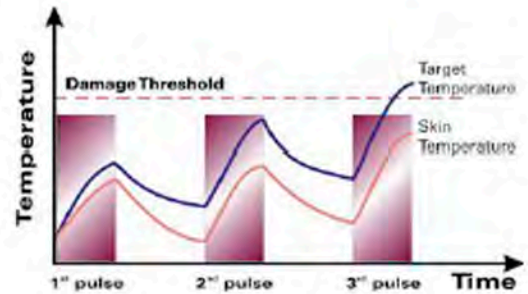

Reduced risk of thermal damage

The advanced integrated skin cooling system of the ThermaeyePlus hand piece prevents excessive temperature increase of the epidermis.

This prevents unwanted side effects on the skin and makes the treatment more tolerable and safety for the patients.

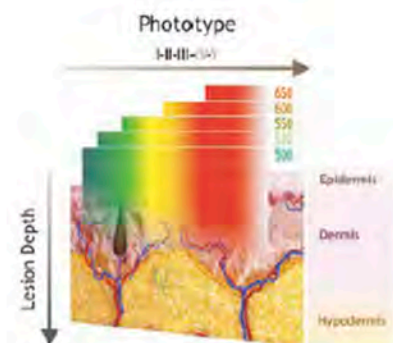

The interaction of the different wavelengths and skin layers determines the choice of the appropriate filter based on the treatment and patient's phototype.

**“Maximum comfort for the patient
Maximum safety for the doctor”**

Incorporated Power meter



ThermaeyePlus features a control tool that is extremely useful in daily practice. The integrated power meter constantly monitors the number of pulses emitted, the real effectiveness of each individual pulse, and the correct operation of the handpiece. By allowing doctors and operators to control the effectiveness and correctness of the energy emitted. Medical studios can thus avoid undue and uncontrolled expenditure: the power meter makes it possible to test the handpiece at all times.



Light, Versatile and User-Friendly

ThermaeyePlus equips interchangeable filters. It is not necessary to change the head: just by replacing the filters, in a few seconds the doctor selects the wavelength that best adapts to the application and pathology of the patient.

The different wavelengths available (520nm, 650nm and SA) in combination with the pulsed technology ensure greater absorption of light by melanin, allowing to work even in the darkest phototypes.

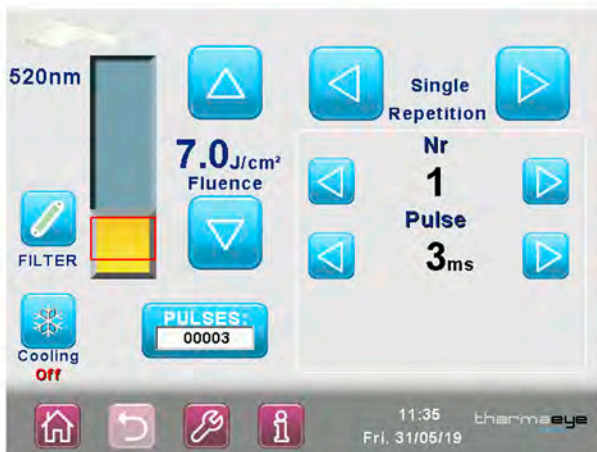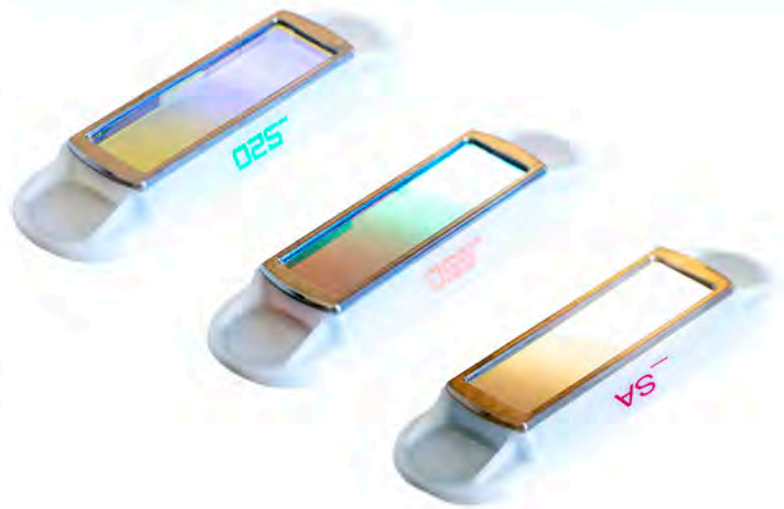

User-friendly Software

ThermaeyePlus has a software designed exclusively for the treatment of meibomian glands and other pathologies associated with dry eye

3

Interchangeable filters, with different spectrum of emission, for a single pulsed light handpiece. It takes only a moment to adjust the emission to the specific application and the patient's phototype.

11

Adjustable levels for contact skin-cooling system integrated in the handpiece.

16Kg

Compact technology. The reduced size of ThermaeyePlus makes it easy to move around.

700gr

Lightweight and compact handpiece for effortless treatment

Power Meter

A power meter incorporated in the system monitors the correct operation of the handpiece at all times.

Technical data

Source	Xe Lamp
Spectrum of Emission	520 — 1200 nm 650 — 1200 nm SA 800 — 1200 nm
Pulse Width	From 3 to 8 ms
Delay Between Pulses	From 5 to 50 ms
Number of Pulses	From 1 to 3
Repetition Rate	2 s (min.)
Treatment Area	48 mm x 13 mm (6.2 cm ²)
Fluence	Up to 25 J/cm ²
Integrated Skin Cooling	Contact Peltier system
Emission Control	Footswitch and/or Fingerswitch
Electrical Requirements	230Vac / 10A (max) 50-60 Hz
Dimensions & Weight	22 cm (H), 45 cm (D), 37 cm (W), 16 Kg

Bibliography

- 1- Prospective evaluation of intense pulsed light and meibomian gland expression efficacy on relieving signs and symptoms of dry eye disease due to meibomian gland dysfunction. Steven J Dell, Ronald N Gaster, Sheila C Barbarino, Derek N Cunningham. Clinical of Ophthalmology, 2 May 2017
- 3- Stimulates circulation and improves inflammation. Ceylan 2004 and Enwemeka 2004.
- 4- Microcoagulation of telangiectasias of palpebral edge and neovascularization surrounding the meibomian glands. Jiang 2016
- 5- Direct antibacterial effect and indirect antibacterial effect by stimulating the emergence of macrophages and bacterial phagocytic activity. Nusbaum 2002, Plaja 2003.
- 6- The role of intense pulsed light (IPL) in the treatment of Meibomian Gland Dysfunction (MGD). Tenna Stefania, Turco Piergiorgio, Brunetti Beniamino, Anotnio DiZazzo, Machi Jacobo, Bonini Stefano, Persichetti Paolo. European Journal of Plastic Surgery 22 May 2019

CE
0123

