	Page		Page
BLEPHAROSTATS	1 - 2	SPECULUMS	1 - 2
RELEVEURS	2	RETRACTORS	2
PLAQUE A PAUPIERES	2	LID PLATE	2
ECARTEUR	2	RETRACTOR	3
COUTEAUX	3	KNIVES	3
AIGUILLE A CORPS ETRANGERS	3	SPUD	3
PORTE-LAMES	3	BLADE BREAKERS	3
TREPANS	4	TREPHINES	4
CURETTES	4	CURETTES	4
SPATULES	5	SPATULAS	5
CROCHETS	5 - 7	HOOKS	5 - 7
DIVERS	8	MISCELLANEOUS	8
CISEAUX	9 - 12	SCISSORS	9 - 12
KR	13	RK	13
CANULES	13 - 18	CANNULAS	13 - 18
PINCES	19 - 24	FORCEPS	19 - 24
PORTE-AIGUILLES	25 - 27	NEEDLE-HOLDERS	25 - 27
BOITES DE STERILISATION	28	STERILIZING TRAYS	28
VOIES LACRYMALES	29	LACRIMAL INSTRUMENTS	29
DCR	30	DACRYOLOGY	30
COUTEAUX DIAMANT HUCO®	31	HUCO® DIAMOND KNIVES	31

CLEANING AND MAINTENANCE OF FCI® SURGICAL INSTRUMENTS

FCI® surgical instruments are shipped NON STERILE.
THEY MUST BE DECONTAMINATED, CLEANED AND STERILIZED BEFORE EACH USE.

Our recommendations for the cleaning of our stainless steel and titanium surgical instruments are as follows:

Rinsing:

- remove blood and tissue immediately after use by rinsing with demineralized or distilled water

Decontamination:

- immerse the instrument in the decontaminating solution and follow the manufacturer's instruction
- avoid the use of chlorine that may corrode the stainless steel instruments

Chemical inactivation of non conventional transmittable agents:

- we recommend that the instruments are soaked in a 1N sodium hydroxide solution prepared with demineralized or distilled water for 60 minutes at 20°C / 66°F. This inactivation should be followed by a very careful soaking, until the pH value is neutral. The presence of residual sodium hydroxide on the instruments may cause corrosion during the sterilization. The presence of chlorine in tap water makes the sodium hydroxide solution corrosive, and therefore it is necessary to use demineralized or distilled water
- rinse the instrument carefully with demineralized water

Cleaning:

- clean the instrument following one of these processes:
 - mild soap solution and soft bristled toothbrush
 - ultrasonic procedure for at least 5 minutes
 - using a washing machine
- dry instrument quickly and completely

Sterilization (physical inactivation of non conventional transmittable agents):

- steam sterilization following a validated cycle according to your institution's standards (recommended parameters are: 18 minutes at 134°C / 293 °F, at 2 bars)
- take care to use an adapted sterilization box

Practical advice:

- treat the instruments made from different metals separately
- do not mix instruments in good condition with instruments that are corroded or damaged
- check the function and the cleanliness of each instrument after cleaning

Cleaning of HUCO® Diamond knives

HUCO® Diamond knives presented on page 31 require a special cleaning procedure. Follow the instructions for use.

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A1-1030

WIRE SPECULUM FOR PHACO
With open blades for unobstructed access to the cataract incision with phaco tip

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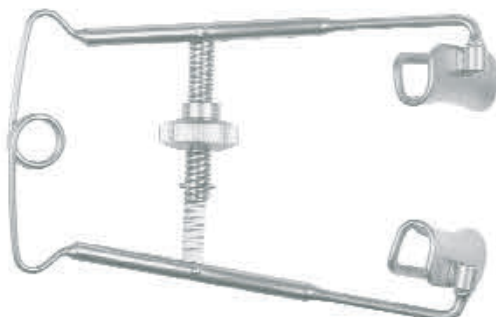
CORNIC NUCLEUS VECTIS
*For small incision surgery
(phacosection)*

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TEYSSIER NUCLEUS BISECTOR
*For manual small incision surgery
(phacosection)*

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GASS ROUCHY HOOK
Retinal detachment hook with hole

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ARNEODO MICRO MANIPULATOR
For phacoemulsification

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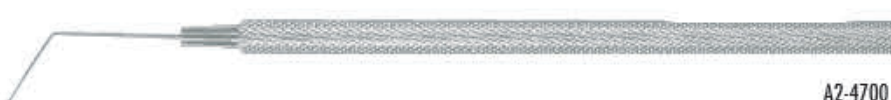
LESTER IOL MANIPULATOR
For lenses with or without holes

A2-4700

A2-4700

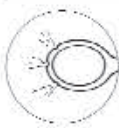
ANSE DE SNELLEN

SNELLEN LENS LOOP

A2-5000

A2-5000

ANSE IRRIGATRICE DE PEARCE

PEARCE IRRIGATING CANNULA

A2-5100

A2-5100

COMPAS DE CASTROVIEJO
20 mm

A2-6000

CASTROVIEJO CALIPER
20 mm spread

A2-6000

SERINGUE 3 ml

A2-7403

SYRINGE 3 ml

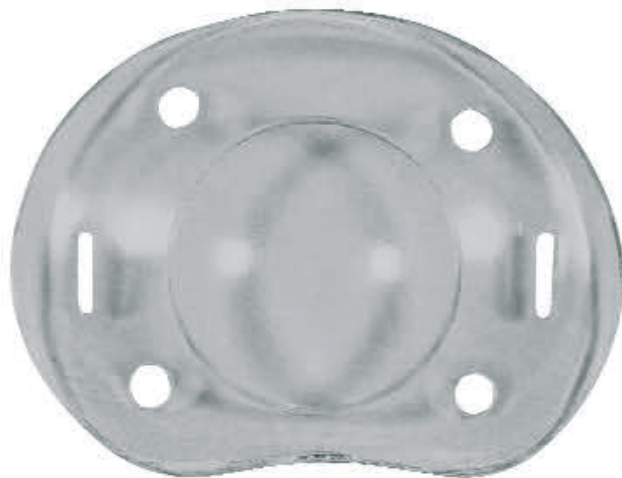
A2-7403

SERINGUE 5 ml

A2-7405

SYRINGE 5 ml

A2-7405

COQUE EN PLASTIQUE
Transparente

A2-7500

UNIVERSAL EYE SHIELD
Transparent

A2-7500

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A2-7710

**SCLERAL DEPRESSOR AND
 FIXATION PICK**
For retinal detachment

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A2-9000

ASTIC-DISC
*Intra-operative instrument for
 qualitative measurement of astigmatism*

A2-9000

CISEAUX DE BONN
9 cm, droit, mousse

A3-1000

BONN SCISSORS
9 cm, straight, blunt tips

A3-1000

CISEAUX DE BONN
9 cm, courbe, mousse

A3-1010

BONN SCISSORS
9 cm, curved, blunt tips

A3-1010

CISEAUX DE BONN
9 cm, droit, pointu

A3-1020

BONN SCISSORS
9 cm, straight, pointed tips

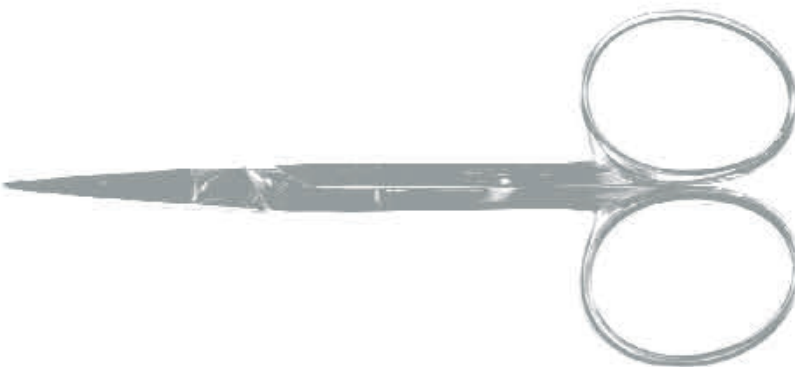
A3-1020

CISEAUX DE BONN
9 cm, courbe, pointu

A3-1030

BONN SCISSORS
9 cm, curved, pointed tips

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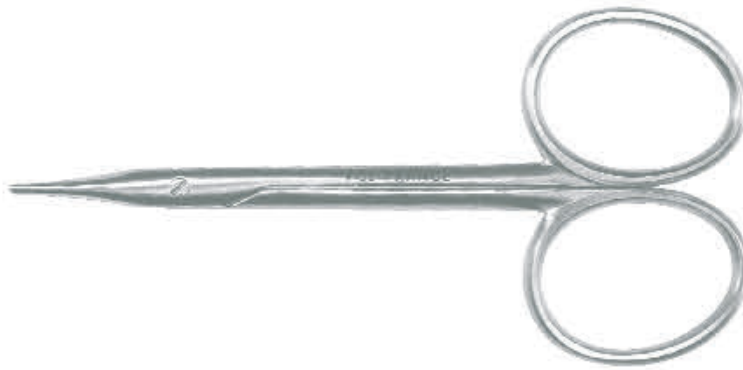
A3-3000

WESTCOTT TENOTOMY SCISSORS
Curved, blunt tips

A3-3000

CISEAUX DE CASTROVIEJO
Lames normales, mousse

A3-3100

CASTROVIEJO SCISSORS
Curved, blunt tips, regular blades

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\varnothing 0,6 mm A5-1235

THX TEYSSIER I/A CANNULA

To remove the cortex during small incision surgery (phacosection)

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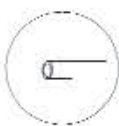
A5-1343

GUERIN CANNULA

To polish the back side of the anterior capsule

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A5-1344

GUERIN CANNULA

To polish the back side of the anterior capsule at twelve o'clock

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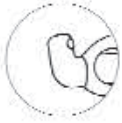
CHAZALON CANNULA
To remove the twelve o'clock cortex going through the side incision
 A5-1460

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CHAZALON CANNULA FOR LEFT HANDED SURGEON
 A5-1465

CANULE THX MODIFIEE LAFORGE
Trou d'aspiration biseauté
Dia. ext. 1,5 mm
 A5-1500

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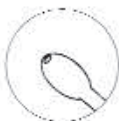
A5-4300

KRATZ ABRASIVE CANNULA
*Olive tip with abrasive inferior side,
front opening*

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A5-4600

THX ABRASIVE CANNULA
Aspirating port facing into curve

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A5-5000

GALEZOWSKI CANNULA
Lacrimal cannula
Small, straight, 14 mm

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A5-6000

MEDIUM GASTAUD CANNULA
For replacing serum with air. For viscosurgery during vitreo retinal surgery

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A5-6010

GASTAUD II CANNULA
To inject liquid perfluoro carbon

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A5-6020

LONG GASTAUD III CANNULA
For aspiration of sub-retinal liquid in myopic eyes

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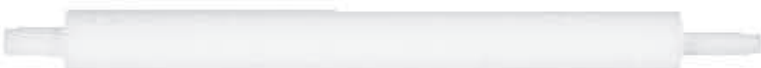
A5-9200

UNIVERSAL I/A HANDPIECE
For interchangeable tips

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A5-9301

STRAIGHT TIP
For A5-9200 handpiece
Aspirating port 0.3 mm

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A5-9303

45° ANGLED TIP
For A5-9200 handpiece
Aspirating port 0.3 mm

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A5-9306

BINKHORST TIP
For A5-9200 handpiece
Front opening

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A5-9400

FCI I/A HANDPIECE
For interchangeable tips

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A5-9401

STRAIGHT INTERCHANGEABLE TIP
For A5-9400 I/A handpiece
Aspirating port 0.3 mm

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A5-9402

CURVED INTERCHANGEABLE TIP
For A5-9400 I/A handpiece
Aspirating port 0.3 mm

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ANGLED INTERCHANGEABLE TIP
For A5-9400 I/A handpiece
Aspirating port 0.3 mm

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A5-9406

BINKHORST INTERCHANGEABLE TIP
For A5-9400 I/A handpiece
Front opening

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A6-1300

**KELMAN McPHERSON
FORCEPS**
*Angled, very delicate 11 mm long
smooth jaws*

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A6-1500

UTRATA FORCEPS
*For capsulorhexis. Very delicate
grasping tips*

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A6-1510

CURVED UTRATA FORCEPS
*For capsulorhexis. Very delicate
grasping tips and extremely thin 11
mm long vaulted shanks*

A6-1510

PINCE DE CORYDON
*Courbe, bouts pointus, corps rond,
titane*

A6-1520

CORYDON FORCEPS
*Extremely thin 11 mm long shanks
cystotome shaped tips (Titanium)*

A6-1520

PINCE DE CORYDON
*Courbe, bouts pointus, corps plat,
titane*

A6-1530

CORYDON FORCEPS
*Extremely thin 11 mm long shanks
cystotome shaped tips (Titanium)
Flat handle*

A6-1530

PINCE D'UTRATA
Bouts mousses, corps plat, titane

A6-1540

UTRATA FORCEPS
*For capsulorhexis. Very delicate
grasping tips and extremely thin
11 mm long vaulted shanks (Titanium)
Flat handle*

A6-1540

PINCE D'UTRATA
Bouts mousses, corps rond, titane

A6-1550

UTRATA FORCEPS
*For capsulorhexis. Very delicate
grasping tips and extremely thin
11 mm long vaulted shanks (Titanium)*

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A6-3100

DRESSING FORCEPS
Serrated, straight

A6-3100

PINCE A DISSEQUER
Droite, 1 x 2 dents (0,7 mm)

A6-3120

TISSUE FORCEPS
Straight, 1 x 2 teeth (0.7 mm)

A6-3120

PINCE D'ADSON BROWN
Mors striés

A6-3150

ADSON BROWN FORCEPS
Serrated jaws, straight

A6-3150

PINCE D'ADSON
1 x 2 dents

A6-3160

ADSON TISSUE FORCEPS
1 x 2 teeth

A6-3160

PINCE DE HALSTED
10 cm, mors striés

A6-4000

HALSTED MOSQUITO FORCEPS
10 cm, serrated jaws

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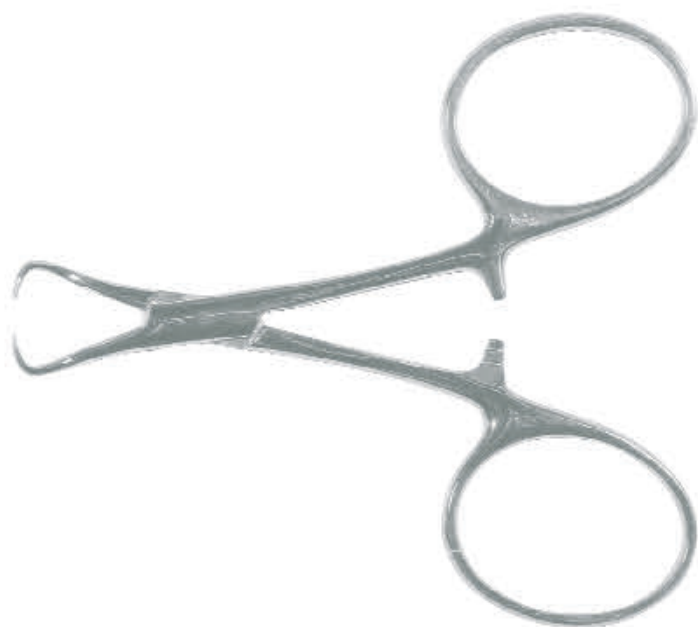
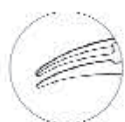
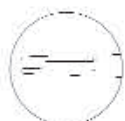
A6-5060

HARMS TUBINGEN FORCEPS
Tying forceps for 7-0 to 9-0 sutures
Curved

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A6-5100

TROUTMAN TYING FORCEPS
Very delicate smooth jaws with long platform for 9-0 to 11-0 sutures
Straight

A6-5100

PINCE A MONOFILAMENT TROUTMAN

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A6-5110

TROUTMAN TYING FORCEPS

*Very delicate smooth jaws with long platform for 9-0 to 11-0 sutures
Curved*

A6-5110

PINCE A MONOFILAMENT
Courbe, titane, corps rond

A6-5120

MONOFILAMENT TYING FORCEPS

*Very delicate smooth jaws with long platform for 9-0 to 11-0 sutures
Curved, titanium*

A6-5120

PINCE A MONOFILAMENT
Courbe, titane, corps plat

A6-5130

MONOFILAMENT TYING FORCEPS

*Very delicate smooth jaws with long platform for 9-0 to 11-0 sutures
Curved, flat handle, titanium*

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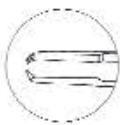
PINCE DE MAC DONALD FINE

A6-9165

McDONALD FOLDABLE FORCEPS
With very fine jaws

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A6-9180

PICK-UP AND FOLD FORCEPS
To pick-up and fold the foldable lens, titanium

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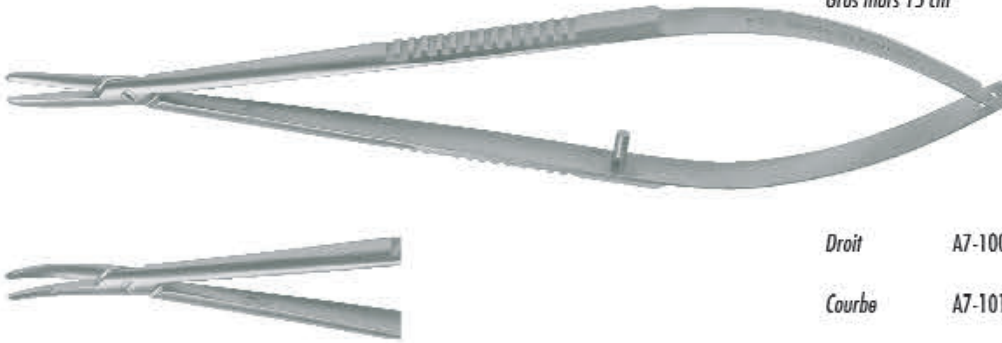
A6-9185

FINE FORCEPS
To grasp the folded optic and insert through the small incision, Titanium

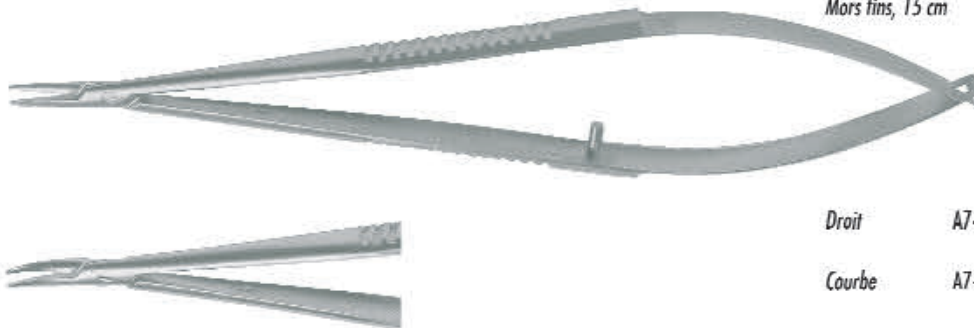
A6-9185

PORTE-AIGUILLE DE CASTROVIEJO
Gros mors 15 cm
CASTROVIEJO NEEDLE HOLDER
Standard jaws, 15 cm

Droit A7-1000

Straight A7-1000

Courbe A7-1010

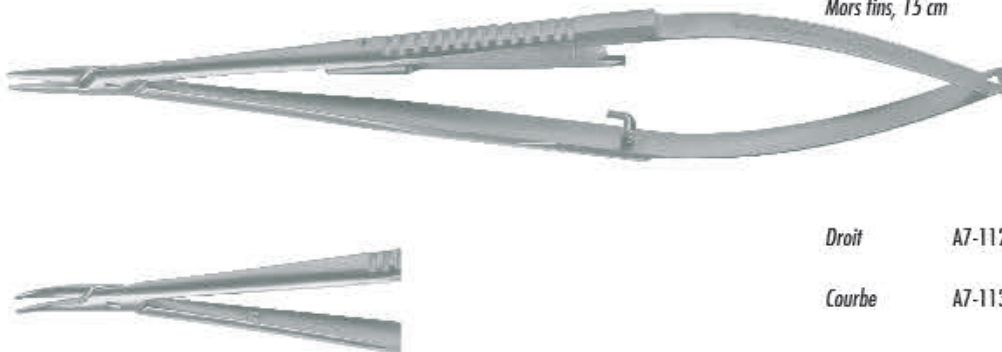
Curved A7-1010

PORTE-AIGUILLE DE CASTROVIEJO
Mors fins, 15 cm
CASTROVIEJO NEEDLE HOLDER
Delicate jaws, 15 cm

Droit A7-1100

Straight A7-1100

Courbe A7-1110

Curved A7-1110

PORTE-AIGUILLE DE CASTROVIEJO A ARRET
Mors fins, 15 cm
CASTROVIEJO NEEDLE HOLDER WITH LOCK
Delicate jaws, 15 cm

Droit A7-1120

Straight A7-1120

Courbe A7-1130

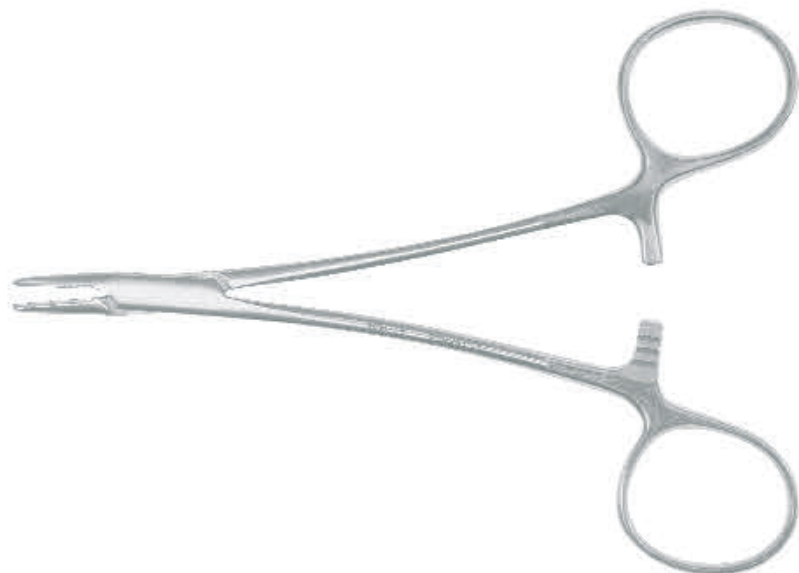
Curved A7-1130

PORTE-AIGUILLE GROS MANCHE
NEEDLE HOLDER LARGE HANDLE

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A7-1400

HALSEY NEEDLE HOLDER
Tungstene jaws

A7-1400

MICRO PORTE-AIGUILLE DE CASTROVIEJO
12 cm, courbe

A7-2010

CASTROVIEJO NEEDLE HOLDER
12 cm long, very delicate jaws, curved

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A7-3010

**BARRAQUER TROUTMAN
NEEDLE HOLDER**
*Short model, extra delicate jaws
Curved*

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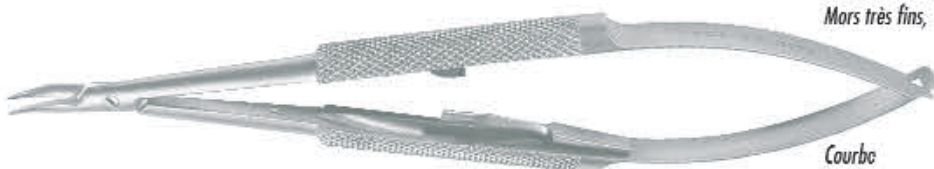
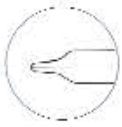
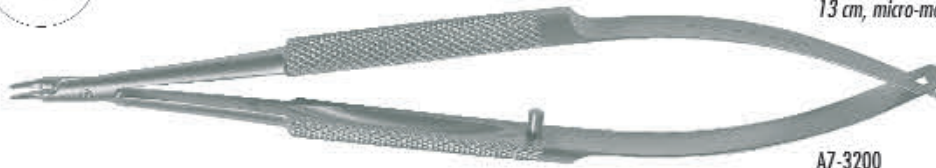
**BARRAQUER NEEDLE HOLDER
WITH LOCK**
Extra delicate jaws, 13 cm
Courbe A7-3130

MICRO PORTE-AIGUILLE DE CORNIC

13 cm, micro-mors becs dégagés

A7-3200

CORNIC NEEDLE HOLDER

Curved, 13 cm, extremely delicate jaws at the tip

A7-3200

MICRO PORTE-AIGUILLE DE CORNIC

13 cm, micro-mors becs dégagés, titane

A7-3210

CORNIC NEEDLE HOLDER

*Curved, 13 cm, extremely delicate jaws at the tip
Titanium*

A7-3210

BOITES DE STERILISATION

STERILIZING TRAYS

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160 x 70 x 20 mm

A8-3000

STERILIZING TRAY

Autoclavable plastic with silicone mat

160 x 70 x 20 mm

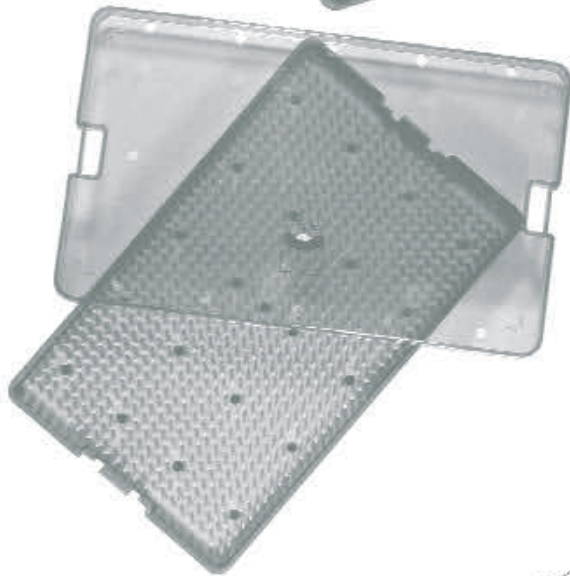
A8-3000

170 x 108 x 20 mm

A8-3003

170 x 108 x 20 mm

A8-3003

260 x 160 x 20 mm

A8-3005

260 x 160 x 20 mm

A8-3005

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260 x 160 x 40 mm

A8-3010

STERILIZING TRAY

Autoclavable plastic with silicone mats
Two levels

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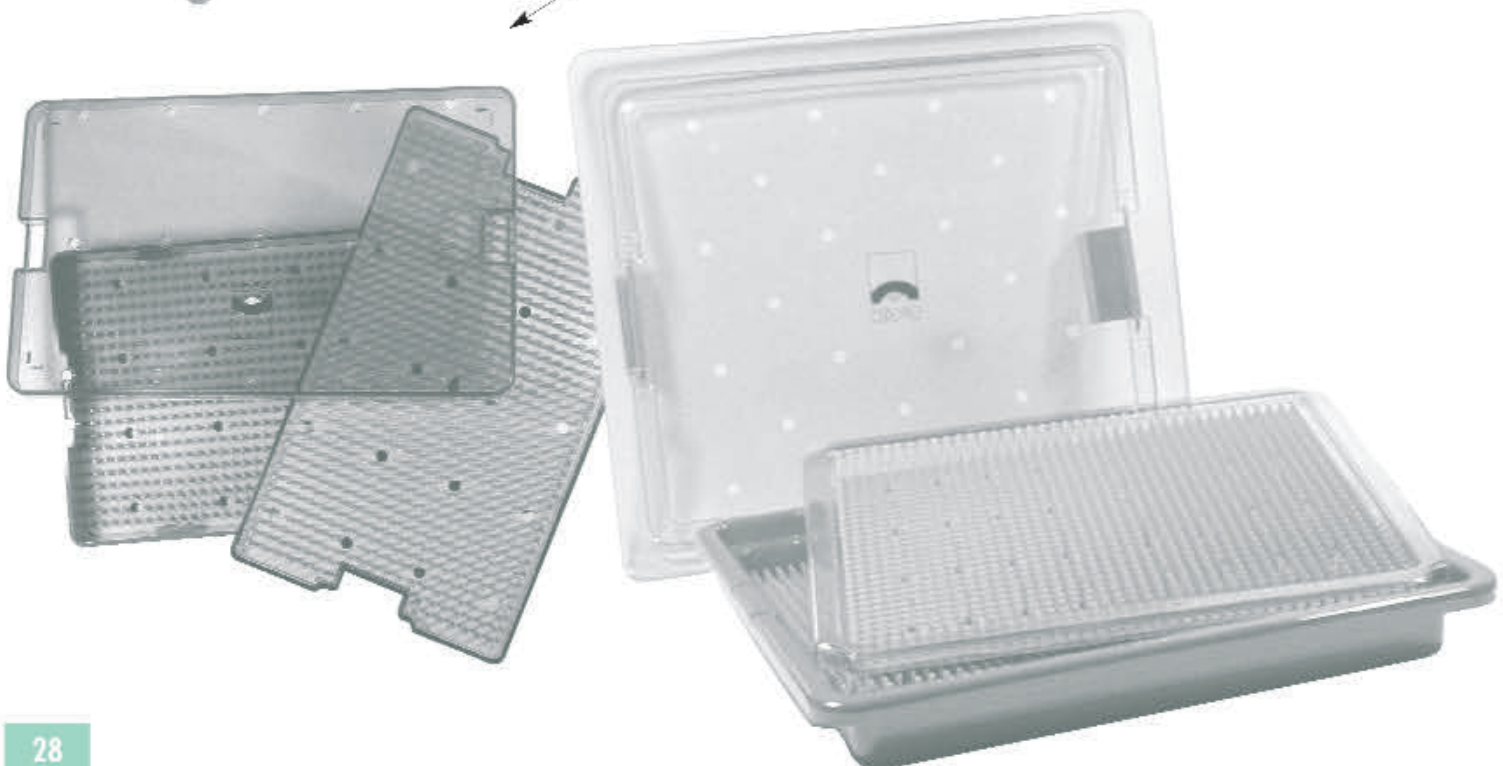
A9-1000

ECARTEUR DE KNAPP
4 griffes mousses

KNAPP LACRIMAL RETRACTOR
4-prong, blunt

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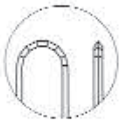
4-1389

**DIAMOND KNIFE «SCLERAL
POCKET» ANGLED**

Crescent shaped blade designed for scleral pocket incision
Wide: 2.0 mm
Length: 4.0 mm
Thickness: 0.17 mm

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A

ABRASIVE, ABRASIVE CANNULA	16
ABRASIVE, CHARLEUX ABRASIVE CANNULA	16
ABRASIVE, KRATZ ABRASIVE CANNULA	16
ABRASIVE, THX ABRASIVE CANNULA	16
ADSON, ADSON TISSUE FORCEPS	20
ADSON BROWN, ADSON BROWN FORCEPS	20
ARNEODO, ARNEODO MICRO MANIPULATOR	7
ARON-ROSA, ARON-ROSA II FORCEPS	24
ARRUGA, ARRUGA HOOK	6
ASTI-DISC	8

B

BARRAQUER, BARRAQUER CANNULA	16
BARRAQUER, BARRAQUER CILIA FORCEPS	20
BARRAQUER, BARRAQUER IRIS SCISSORS	11
BARRAQUER, BARRAQUER NEEDLE HOLDER	26
BARRAQUER, BARRAQUER SPATULA	5
BARRAQUER, BARRAQUER WIRE SPECULUM	1
BARRAQUER COLIBRI, BARRAQUER COLIBRI FORCEPS	23
BARRAQUER TROUTMAN, BARRAQUER TROUTMAN NEEDLE HOLDER	26
BERNARD, J.A. BERNARD LACRIMAL CURETTE	4
BINKHORST, BINKHORST CANNULA	13
BISECTOR, TEYSSIER NUCLEUS BISECTOR	5
BLADE BREAKER, TROUTMAN BLADE BREAKER AND HOLDER	3
BONN, BONN FORCEPS	22,23
BONN, BONN MICRO IRIS HOOK	6
BONN, BONN RETRACTOR	2
BONN, BONN SCISSORS	9

C

CALIPER, CASTROVIEJO CALIPER	8
CANNULA, ABRASIVE CANNULA	16
CANNULA, BARRAQUER CANNULA	16
CANNULA, BINKHORST CANNULA	13
CANNULA, CHARLEUX ABRASIVE CANNULA	16
CANNULA, CHARLEUX CANNULA	13
CANNULA, CHAZALON CANNULA	15
CANNULA, GALEZOWSKI CANNULA	16
CANNULA, GASTAUD II CANNULA	17
CANNULA, GUERIN CANNULA	14
CANNULA, HEALON CANNULA	15
CANNULA, HYDRO-DISSECTION CANNULA	15
CANNULA, IRRIGATING CANNULA	14
CANNULA, KRATZ ABRASIVE CANNULA	16
CANNULA, LAFORGE THX CANNULA	15
CANNULA, LONG GASTAUD III CANNULA	17
CANNULA, MEDIUM GASTAUD CANNULA	17
CANNULA, ORIGINAL SIMCOE I/A CANNULA	15
CANNULA, PEARCE IRRIGATING CANNULA	7
CANNULA, REVERSED SIMCOE I/A CANNULA	15
CANNULA, RYCRÖFT AIR CANNULA	16
CANNULA, SIMCOE CANNULA	13
CANNULA, THX ABRASIVE CANNULA	16
CANNULA, THX CLUB DE GOLF I/A CANNULA	15
CANNULA, THX I/A CANNULA	14
CANNULA, THX TEYSSIER I/A CANNULA	14
CANNULA, THX TURUT I/A CANNULA	14
CASTROVIEJO, CASTROVIEJO CALIPER	8
CASTROVIEJO, CASTROVIEJO EYE SPECULUM	2
CASTROVIEJO, CASTROVIEJO NEEDLE HOLDER	25,26
CASTROVIEJO, CASTROVIEJO SCISSORS	11
CASTROVIEJO, CASTROVIEJO SUTURING FORCEPS	23
CHALAZION FORCEPS, DESMARRS CHALAZION FORCEPS	20
CHARLEUX, CHARLEUX ABRASIVE CANNULA	16
CHARLEUX, CHARLEUX CANNULA	13
CHAZALON, CHAZALON CANNULA	15
CHOP, PHACO CHOP	7
CILIA FORCEPS, BARRAQUER CILIA FORCEPS	20
CLEAR CORNEA, DIAMOND KNIFE "CLEAR CORNEA"	31
COLIBRI, COLIBRI FORCEPS	22
COLLAR BUTTON, COLLAR BUTTON HOOK	6
CONNECTOR, MALE-MALE CONNECTOR	14
CORCELLE, CORCELLE EYE SPECULUM	1
CORNIC, CORNIC CAPSULOTOMY SCISSORS	12
CORNIC, CORNIC FORCEPS	23
CORNIC, CORNIC NEEDLE HOLDE	27
CORNIC, CORNIC NUCLEUS VECTIS	5
CORYDON, CORYDON FORCEPS	19
CULLER, CULLER IRIS SPATULA	5
CURETTE, J.A. BERNARD LACRIMAL CURETTE	4
CURETTE, MEYERHOFFER CHALAZION CURETTE	4

D

DESMARRS, DESMARRS CHALAZION FORCEPS	20
DESMARRS, DESMARRS LID RETRACTOR	2

DIAMOND KNIFE, 3 STEPS DIAMOND KNIFE	31
DIAMOND KNIFE, DIAMOND KNIFE "CLEAR CORNEA"	31
DIAMOND KNIFE, DIAMOND KNIFE "SCLERAL POCKET"	31
DIAMOND KNIFE, RK DIAMOND MICROMETER KNIFE	31
DILATOR, DOUBLE END LACRIMAL DILATOR	29
DILATOR, SICHEL WILDER LACRIMAL DILATOR	29
DRESSING, DRESSING FORCEPS	20

E

ELSCHNIG, ELSCHNIG SUPERIOR RECTUS FORCEPS	22
ENUCLEATION, ENUCLEATION SCISSORS	10
EVISCERATION SPOON	4

F

FINE, FINE FORCEPS	24
FOLDABLE LENS, GRASPING FOLDABLE LENS FORCEPS	24
FORCEPS, ADSON BROWN FORCEPS	20
FORCEPS, ADSON TISSUE FORCEPS	20
FORCEPS, ARON-ROSA II FORCEPS	24
FORCEPS, BARRAQUER CILIA FORCEPS	20
FORCEPS, BARRAQUER COLIBRI FORCEPS	23
FORCEPS, BONN FORCEPS	22,23
FORCEPS, CASTROVIEJO SUTURING FORCEPS	23
FORCEPS, COLIBRI FORCEPS	22
FORCEPS, CORNIC FORCEPS	23
FORCEPS, CORYDON FORCEPS	19
FORCEPS, CURVED UTRATA FORCEPS	19
FORCEPS, DEMARRS CHALAZION FORCEPS	20
FORCEPS, DRESSING FORCEPS	20
FORCEPS, ELSCHNIG SUPERIOR RECTUS FORCEPS	22
FORCEPS, FINE FORCEPS	24
FORCEPS, GRASPING FOLDABLE LENS FORCEPS	24
FORCEPS, GREEN FIXATION FORCEPS	22
FORCEPS, HALSTED MOSQUITO FORCEPS	21
FORCEPS, HARMS TUBINGEN FORCEPS	21
FORCEPS, KELMAN McPHERSON FORCEPS	19
FORCEPS, McDONALD FOLDABLE FORCEPS	24
FORCEPS, McDONALD II FOLDABLE FORCEPS	24
FORCEPS, MONOFILAMENT TYING FORCEPS	22
FORCEPS, PAUFIQUE FORCEPS	23
FORCEPS, PICK-UP AND FOLD FORCEPS	24
FORCEPS, TISSUE FORCEPS	20
FORCEPS, TROUTMAN TYING FORCEPS	21,22
FORCEPS, UTRATA FORCEPS	19
FOREIGN BODY SPUD	3

G

GALAND, GALAND CAPSULOTOMY SCISSORS	12
GALEZOWSKI, GALEZOWSKI CANNULA	16
GALEZOWSKI, GALEZOWSKI LACRIMAL PROBE	29
GASS ROUCHY, GASS ROUCHY HOOK	5
GASTAUD, GASTAUD II CANNULA	17
GASTAUD, LONG GASTAUD III CANNULA	17
GASTAUD, MEDIUM GASTAUD CANNULA	17
GAUGE, HUCO KREMER BLADE GAUGE	31
GILLIES, GILLIES LID HOOK	6
GRAEFE, GRAEFE MUSCLE HOOK	5
GREEN, GREEN FIXATION FORCEPS	22
GUERIN, GUERIN CANNULA	14

H

HALSEY, HALSEY NEEDLE HOLDER	26
HALSTED, HALSTED MOSQUITO FORCEPS	21
HANDLE, KELLER CANNULA HANDLE	17
HANDLE, SHORT KELLER CANNULA HANDLE	17
HANDPIECE, FCI I/A HANDPIECE	18
HANDPIECE, UNIVERSAL I/A HANDPIECE	18
HARMS TUBINGEN, HARMS TUBINGEN FORCEPS	21
HEALON, HEALON CANNULA	15
HOFFER, HOFFER OPTIC ZONE MARKER	13
HOOK, ARRUGA HOOK	6
HOOK, BONN MICRO IRIS HOOK	6
HOOK, COLLAR BUTTON HOOK	6
HOOK, GASS ROUCHY HOOK	5
HOOK, GILLIES LID HOOK	6
HOOK, GRAEFE MUSCLE HOOK	5
HOOK, JAFFE BECHERT HOOK	7
HOOK, RETINAL DETACHMENT HOOK	5
HOOK, SINSKEY CORNIC HOOK	6
HOOK, SINSKEY HOOK	6
HUCO KREMER, HUCO KREMER BLADE GAUGE	31
HYDRO-DISSECTION, HYDRO-DISSECTION CANNULA	15

J

JÄGER, JÄGER LID PLATE	2
JAFFE BECHERT, JAFFE BECHERT HOOK	7

K

KATZIN, KATZIN CORNEAL TRANSPLANT SCISSORS	11
KELLER, KELLER CANNULA HANDLE	17
KELLER, SHORT KELLER CANNULA HANDLE	17
KELMAN McPHERSON, KELMAN McPHERSON FORCEPS	19
KERRISON, KERRISON BONE NIBBLING RONGEUR	30
KNAPP, KNAPP LACRIMAL RETRACTOR	30
KRATZ, KRATZ ABRASIVE CANNULA	16

L

LAFORGE CORNIC, LAFORGE CORNIC CAPSULOTOMY SCISSORS	12
LAFORGE THX, LAFORGE THX CANNULA	15
LEBUISSON, LEBUISSON MICRO MANIPULATOR	7
LESCURE GALAND, LESCURE GALAND CAPSULOTOMY SCISSORS	12
LESTER, LESTER IOL MANIPULATOR	7
LID PLATE, JAEGER LID PLATE	2
LOOP, SNELLEN LENS LOOP	7

M

MANIPULATOR, ARNEODO MICRO MANIPULATOR	7
MANIPULATOR, LEBUISSON MICRO MANIPULATOR	7
MANIPULATOR, LESTER IOL MANIPULATOR	7
MARKER, HOFFER OPTIC ZONE MARKER	13
McDONALD, McDONALD FOLDABLE FORCEPS	24
McDONALD II, McDONALD II FOLDABLE FORCEPS	24
MEYERHOEFER, MEYERHOEFER CHALAZION CURETTE	4
MONOFILAMENT, MONOFILAMENT TYING FORCEPS	22
MOSQUITO, HALSTED MOSQUITO FORCEPS	21

N

NEEDLE HOLDER, BARRAQUER NEEDLE HOLDER	26
NEEDLE HOLDER, BARRAQUER TROUTMAN NEEDLE HOLDER	26
NEEDLE HOLDER, CASTROVIEJO NEEDLE HOLDER	25,26
NEEDLE HOLDER, CORNIC NEEDLE HOLDER	27
NEEDLE HOLDER, HALSEY NEEDLE HOLDER	26
NEEDLE HOLDER, NEEDLE HOLDER LARGE HANDLE	25

P

PARKER, PARKER BLADE # 11	3
PARKER, PARKER HANDLE # 3	3
PAUFIQUE, PAUFIQUE FORCEPS	23
PAUFIQUE, PAUFIQUE SCLEROTOME KNIFE	3
PEARCE, PEARCE IRRIGATING CANNULA	7
PHACO, WIRE SPECULUM FOR PHACO	1
PICK-UP AND FOLD, PICK-UP AND FOLD FORCEPS	24
PLEY, PLEY EYE SPECULUM	1
PROBE, GALEZOWSKI LACRIMAL PROBE	29

R

RAZOR BLADES	3
RETRACTOR, BONN RETRACTOR	2
RETRACTOR, DESMARRES LID RETRACTOR	2
RETRACTOR, KNAPP LACRIMAL RETRACTOR	30
RING, GLOBE FIXATION RING	13
RK, RK DIAMOND MICROMETER KNIFE	31
RONGEUR, KERRISON BONE NIBBLING RONGEUR	30
RYCROFT, RYCROFT AIR CANNULA	16

S

SALZ, SALZ NUCLEUS SPLITTER	24
SCISSORS, BARRAQUER IRIS SCISSORS	11
SCISSORS, BONN SCISSORS	9
SCISSORS, CASTROVIEJO SCISSORS	11
SCISSORS, CORNIC CAPSULOTOMY SCISSORS	12
SCISSORS, ENUCLEATION SCISSORS	10
SCISSORS, EYE SCISSORS	9
SCISSORS, GALAND CAPSULOTOMY SCISSORS	12
SCISSORS, KATZIN CORNEAL TRANSPLANT SCISSORS	11
SCISSORS, LAFORGE CORNIC CAPSULOTOMY SCISSORS	12

SCISSORS, LESCURE GALAND CAPSULOTOMY SCISSORS	12
SCISSORS, SEVRIN STEVENS TENOTOMY SCISSORS	10
SCISSORS, VANNAS IRIS SCISSORS	12
SCISSORS, VANNAS SCISSORS	12
SCISSORS, WECKER IRIS SCISSORS	10
SCISSORS, WESTCOTT TENOTOMY SCISSORS	11
SCLERAL DEPRESSOR, SCLERAL DEPRESSOR AND FIXATION PICK	8
SCLERAL POCKET, DIAMOND KNIFE "SCLERAL POCKET"	31
SCLEROTOME, PAUFIQUE SCLEROTOME KNIFE	3
SERREFINE	21
SEVRIN STEVENS, SEVRIN STEVENS TENOTOMY SCISSORS	10
SICHEL WILDER, SICHEL WILDER LACRIMAL DILATOR	29
SIMCOE, ORIGINAL SIMCOE I/A CANNULA	15
SIMCOE, REVERSED SIMCOE I/A CANNULA	15
SIMCOE, SIMCOE CANNULA	13
SINSKEY, SINSKEY HOOK	6
SINSKEY CORNIC, SINSKEY CORNIC HOOK	6
SNELLEN, SNELLEN LENS LOOP	7
SPATULA, BARRAQUER SPATULA	5
SPATULA, CULLER IRIS SPATULA	5
SPECULUM, BARRAQUER WIRE SPECULUM	1
SPECULUM, CASTROVIEJO EYE SPECULUM	2
SPECULUM, CORCELLE EYE SPECULUM	1
SPECULUM, PLEY EYE SPECULUM	1
SPECULUM, WIRE SPECULUM FOR PHACO	1
SPLITTER, SALZ NUCLEUS SPLITTER	24
SPOON, EVISCERATION SPOON	4
SPUD, FOREIGN BODY SPUD	3
STERILIZING TRAYS	28
SYRINGE	8

T

TEYSSIER, TEYSSIER NUCLEUS BISECTOR	5
THX, THX ABRASIVE CANNULA	16
THX, THX CLUB DE GOLF I/A CANNULA	15
THX, THX I/A CANNULA	14
THX TEYSSIER, THX TEYSSIER I/A CANNULA	14
THX TURUT, THX TURUT I/A CANNULA	14
TIP, 45° ANGLED INTERCHANGEABLE TIP	18
TIP, 45° ANGLED TIP	18
TIP, 90° ANGLED INTERCHANGEABLE TIP	18
TIP, 90° ANGLED TIP	18
TIP, BINKHORST INTERCHANGEABLE TIP	18
TIP, BINKHORST TIP	18
TIP, CURVED INTERCHANGEABLE TIP	18
TIP, STRAIGHT INTERCHANGEABLE TIP	18
TIP, STRAIGHT TIP	18
TISSUE, ADSON TISSUE FORCEPS	20
TISSUE, TISSUE FORCEPS	20
TOWEL CLAMP	21
TRAYS, STERILIZING TRAYS	28
TREPHINE, TREPHINE FCI TYP	4
TROUTMAN, BLADE BREAKER AND HOLDER	3
TROUTMAN, TROUTMAN TYING FORCEPS	21,22
TUBING, SILICONE TUBING	15

U

UNIVERSAL EYE SHIELD	8
UTRATA, CURVED UTRATA FORCEPS	19
UTRATA, UTRATA FORCEPS	19

V

VANNAS, VANNAS SCISSORS	12
VANNAS, VANNAS IRIS SCISSORS	12
VECTIS, CORNIC NUCLEUS VECTIS	5

W

WECKER, WECKER IRIS SCISSORS	10
WESTCOTT, WESTCOTT TENOTOMY SCISSORS	11

20 - 22, RUE LOUIS ARMAND
75015 PARIS

TEL : +33 (0) 1 53 98 98 98 FAX : +33 (0) 1 53 98 98 99
E-mail : fci@fci.fr Web : <http://www.fci.fr>