



SCIENTIFICALLY PROVEN,
NATURAL DRY EYE RELIEF

TABLE OF CONTENTS

Introduction 2

MOIST-HEAT THERAPIES / COLD THERAPIES 5

D.E.R.M.™ 7

Wellness Kits 9

tranquileyes® 11

tranquileyes xl® 16

Stye Kit™ 18

eyecloud® 19

NIGHTTIME DRY EYE RELIEF 21

eyeseals™ 23

eyeseals™ 4.0..... 23

Onyx™ and Quartz™ 24

tranquileyes® 25

Assuring the best fit..... 27

EYELID AND FACIAL THERAPIES 28

Tea Tree Eyelid and Facial Cleansers 29

Essential Nine™ 31

Facial Hydrating Sprays..... 32

DAYTIME DRY EYE RELIEF 33

Moisture Release Eyewear™ 33

Dry Eye Protocols..... 35

Drop-Ship Program for Patients..... 37



CHANGING LIVES
SINCE 2003

If you suffer from dry, irritated or allergy-prone eyes, you're not alone. An estimated 30 million Americans report symptoms consistent with Dry Eye. The impact on your life can range from mild symptoms (like feeling a foreign body sensation in your eye), to severe symptoms (like the inability to wear contact lenses, nights of lost sleep, blurred vision, and constant irritation). Severe Dry Eye disease can affect your vision, concentration, health, and relationships.

At Eye Eco, we've dedicated ourselves to providing relief to the millions of Americans

who suffer from mild, moderate and severe dry eye. We offer natural, drug-free, and scientifically proven products that let you live your daily life without having to think about your Dry Eye.

OUR STORY

Eye Eco founder, Suzanne Paulson, was attending a consumer product review in 2002 when she learned about Sjögren's syndrome, a chronic condition in which white blood cells attack and destroy the glands in the body that produce tears. Suzanne was moved by the patient stories and believed that she could help with one of the worst symptoms of the condition, severe Dry Eye. Using her 15 years of eye care knowledge and industry experience, Suzanne founded Eye Eco and began to pioneer solutions that have since helped thousands of Sjögren's syndrome and other Dry Eye patients.

At Eye Eco, we believe eye health requires a holistic approach. Recent research has shown

that many eye problems may be prevented through a long-term daily regimen. Aside from helping patients who have Dry Eye now, we want to spread the word that even patients that currently do not suffer from Dry Eye can take steps to protect their eyes and help prevent chronic Dry Eye from developing prematurely.

tranquileyes®

Eye Eco's first product, tranquileyes®, was the first all-natural, drug-free treatment for dry eyes designed to work in tandem with other hydration aids (artificial tears). The response by patients was overwhelmingly positive. Since then, Suzanne and her team have worked to improve the original product, as well as develop further innovations in the therapeutic eye product field. Now, Eye Eco therapies for Dry Eye management are tested and recommended by thousands of eye doctors and are being used by tens of thousands of patients worldwide.



SCIENTIFICALLY PROVEN NATURAL PRODUCTS FOR DRY EYE RELIEF

OUR PROMISE

We at Eye Eco are proud to help Dry Eye patients find relief. All of our products are backed by a satisfaction guarantee. Please feel free to call us with any questions, comments or ideas. We would love to hear from you!

THE COMPANY

Eye Eco Inc., is a California-based company committed to developing innovative, natural and scientifically proven Dry Eye relief products and therapies.

We are also a company dedicated to giving back. Since day one, we have donated a percentage of our sales to the Sjögren's Syndrome Foundation. We also give back to InfantSEE® and distribute wildflower seeds to help save the bees. Visit our website for more information on these important causes.

DRY EYE: ARE YOU AT RISK?

Many factors contribute to the development of Dry Eye including lifestyle, environment and physical condition. Understanding the causes is important to managing existing symptoms and in prevention. Chronic Dry Eye can significantly impact the quality of both home and work life. Some of the most common causes include:

AGE. Every day, more than 10,000 Americans reach age 55. Older patients are at an increased risk for Dry Eye.

GENDER. Women are twice as likely to develop Dry Eye than men. Hormone changes are one key reason for this finding.

LIFESTYLE. It is estimated people spend over 12 hours a day staring at a computer, tablet or phone daily. Any staring activity can contribute significantly to Dry Eye as you forget to blink.

ENVIRONMENT. This includes high altitude, airplane travel, air conditioning, forced air heating, wind, dust and smoke.

LID MARGIN DISEASE. This is especially true of Meibomian gland dysfunction and blepharitis.

MEDICATIONS. Some over-the-counter medications such as antidepressants, decongestants and oral antihistamines can contribute to Dry Eye symptoms.

MEDICAL AND SYSTEMIC CONDITIONS.

This includes Sjögren's syndrome, diabetes, rheumatoid arthritis and thyroid disease. Sleep apnea patients who use CPAP devices are also at risk for Dry Eye.

CONTACT LENS WEAR. While certain lens types may be better suited for Dry Eye sufferers, contact lens wear and products associated with contact lens wear can contribute to Dry Eye. Dry Eye management is important to successful contact lens wear.

OCULAR SURGERY. Surgical procedures, such as LASIK or cataract removal, can cause Dry Eye in patients with risk factors. For best surgical results, treat Dry Eye symptoms prior to surgery and manage any temporary Dry Eye following surgery.

Be sure to tell your eye doctor if you have any of these risk factors for Dry Eye disease.

SCIENTIFICALLY PROVEN NATURAL PRODUCTS FOR DRY EYE RELIEF

CONTROLLED MOIST-HEAT FOR MILD TO SEVERE DRY EYE RELIEF

tranquileyes® tranquileyes xl®
stye kit™ der.m™ eyecloud.

COLD THERAPY TO REDUCE PUFFINESS AND ALLERGY SYMPTOMS

tranquileyes® tranquileyes xl®
der.m™ eyecloud.

HYDRATING EYELID AND FACIAL THERAPY

FacialHydrating™ SPRAY E9 EYELID & SKIN THERAPY
TeaTree
EYELID & FACIAL
CLEANSER

DAYTIME DRY EYE RELIEF FOR ALLERGIES, CONTACT LENS WEAR, AQUEOUS DEFICIENT DRY EYE

moisture release
EYEWEAR™

NIGHTTIME DRY EYE RELIEF FOR LAGOPHTHALMOS, COMPROMISED LID SEAL, CPAP USE

tranquileyes®
eyeseals™ eyeseals 4.0
ONIX™ QUARTZ™

THE SCIENCE OF MOIST-HEAT THERAPY

DRY EYE THERAPIES: WHY APPLYING MOIST-HEAT WORKS

For decades, eye doctors have recommended various home remedies to help relieve Dry Eye symptoms. These have included applying a warm wet wash cloth, rice in a sock, hot potato, or even a boiled egg over the eyelid margin for temporary relief! While well-intended, these methods have proven ineffective for long-term relief.¹ Now we know why...

Extensive research² during the past few years has shown that to manage a variety of mild to severe ocular surface conditions, including Dry Eye, MGD (Meibomian Gland Dysfunction) and blepharitis, a moist heat compress needs to sustain temperatures between 102° and 108° Fahrenheit for up to 20 minutes. Moist heat can also help prevent the premature development of ocular surface conditions, which can lead to discomfort and compromised vision.


In a healthy eye, the Meibomian glands secrete oil every time you blink. In the most typical type of Dry Eye, which is MGD, the critical oils become clogged. When this occurs, the fluid coming out of the

glands becomes the consistency of toothpaste instead of the desirable consistency of olive oil. Moist-heat therapy stimulates tear production by promoting circulation, speeding heat transfer, and loosening the oils so that they return to their healthy state. Additionally, the humidity associated with moist-heat therapy stabilizes the tear film and hydrates the sensitive eyelid and surrounding skin.^{3,4,5}

THE KEY TO AN EFFECTIVE MOIST-HEAT THERAPY IS IN THE SCIENCE

Approximately 50% of people with Dry Eye or MGD have a mild condition which requires 5 minutes of controlled temperature to relieve their symptoms. Moderate to severe patients require 10-20 minutes. An eye doctor specializing in Dry Eye can help you determine what type of Dry Eye you have.

HISTORIC HOME REMEDIES

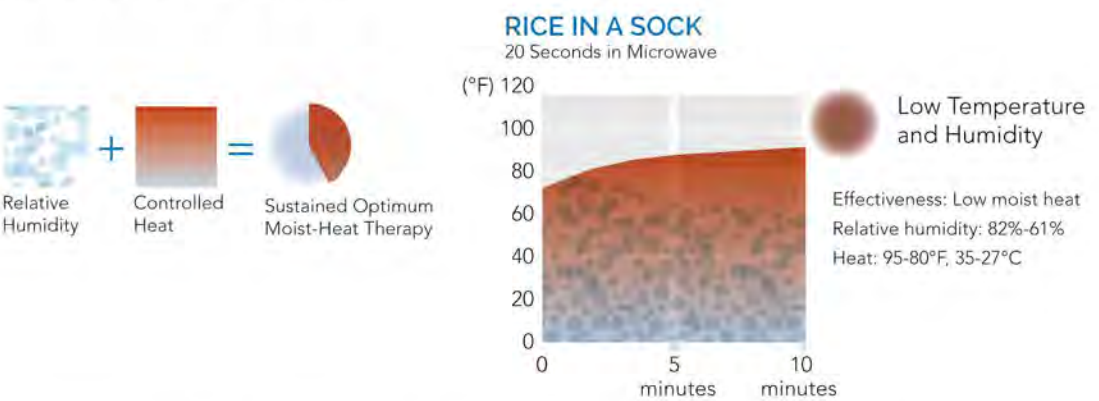

¹Blackie, C. A., Korb, D. R., Schubert, J. R., Murakami, D. K., & FAAO, O. M. (2015). Using warm compresses to treat meibomian gland disease.
²Geerling, G., Tauber, J., Baudouin, C., Goto, E., Matsumoto, Y., O'Brien, T., ... & Nichols, K. K. (2011). The international workshop on meibomian gland dysfunction: report of the subcommittee on management and treatment of meibomian gland dysfunction. Investigative ophthalmology & visual science, 52(4), 2050-2064.
³Blackie, C. A., Solomon, J. D., Greiner, J. V., Holmes, M., & Korb, D. R. (2008). Inner eyelid surface temperature as a function of warm compress methodology. Optometry & Vision Science, 85(8), 675-683.
⁴Olson, M. C., Korb, D. R., & Greiner, J. V. (2003). Increase in tear film lipid layer thickness following treatment with warm compresses in patients with meibomian gland dysfunction. Eye & contact lens, 29(2), 96-99
⁵Korb, D. R., Greiner, J. V., Glonek, T., Esbah, R., Finnemore, V. M., & Whalen, A. C. (1996). Effect of periocular humidity on the tear film lipid layer. Cornea, 15(2), 129-134.

NATURAL, SCIENTIFICALLY PROVEN MOIST-HEAT

EYE ECO DIFFERENCE – HOW CAN YOU TELL IT'S MOIST HEAT? TEST IT.

Eye Eco offers three scientifically proven moist-heat therapies which deliver 5-25 minutes of controlled temperature and natural humidity for immediate and long-term Dry Eye relief. Why three? Simple. Dry Eye patients differ and so do their moist-heat therapy needs.

Using hygrometers, we measure the temperature and relative humidity of our technologies. This testing assures that patients receive safe, controlled moist-heat therapies which deliver the optimum performance for their condition. We recognize this evaluative process as essential to ensure our technologies do not exceed desired temperature levels, potentially burning or drying out the sensitive eyelid and surrounding skin.

MOIST HEAT VS. DRY HEAT

The eyelid is the thinnest skin in the human body. Research has indicated that if you have Dry Eye, you might also have dry skin. We recommend always applying moist heat to the lid margin, never dry heat.

MOIST HEAT	DRY HEAT
Speeds heat transfer	Pulls moisture away from the eyelid and sensitive surrounding skin
Loosens the oils in Meibomian glands	Decreases relative humidity
Increases relative humidity to help stabilize tear film	Increases instability of tear film
Slows tear evaporation	Prematurely ages the skin
Promotes circulation	
Penetrates deeply, hydrating the eyelid and surrounding skin	





● Shown: D.E.R.M.™ with cotton liner

MILD DRY EYE THERAPY



TREATING MILD DRY EYE WITH D.E.R.M.™

Perfect for mild Dry Eye patients, the Dry Eye Relief Mask, D.E.R.M.™, delivers 5 to 9 minutes of moist heat safely and naturally. An affordable solution for mild Dry Eye, D.E.R.M.™ is the entry-level moist-heat compress option preferred by eye doctors and patients around the globe.

The mask contains silica beads which naturally absorb moisture from the air and release it when microwaved. Simply heat for 20 seconds on high to receive 5 to 6 minutes of soothing moist heat.

D.E.R.M.™ comes with complimentary optional liners which can be moistened and placed on the back of the mask prior to heating. These liners create a moisture barrier, safely allowing you to extend the moist-heat treatment for 7 to 9 minutes.

D.E.R.M.™ can also be placed in the freezer for cold therapies to relieve symptoms of allergies, sinus pressure and puffiness.

Our new microfiber Comfort Cover™ will keep the mask clean and feels great against the face. It can be used with or without the liners. Available only through eyecare professionals, ask your eye doctor for the D.E.R.M.™ today!





D.E.R.M.™ Dry Eye Relief Mask



RECOMMENDED FOR

Mild Dry Eye

▶ Watch the D.E.R.M.™ video at www.eyeeeco.com

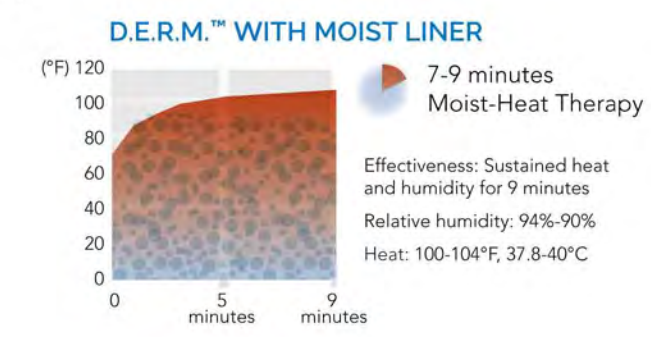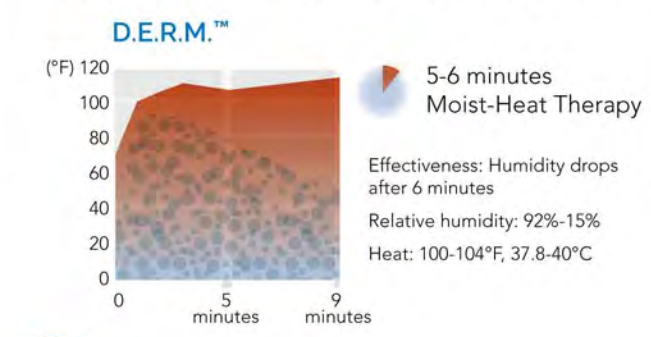

ACCESSORIES



Eye Eco Comfort Cover™

Introducing the new Eye Eco Comfort Cover™! Made of soft microfiber, it helps keep the mask clean, prolonging the life of the mask.



Disposable liners are available in packs of 100.



D.E.R.M.™ WELLNESS KITS

MILD DRY EYE THERAPY

Keeping your eyes and eyelids looking and feeling their best has never been so easy! Whether you are managing mild Dry Eye or proactively preventing it from occurring, our new D.E.R.M.™ Wellness Kits are a great option. Each kit includes one D.E.R.M.™ mask (with or without Comfort Cover™), one of our three Tea Tree Eyelid & Facial Cleansers, and either a soothing Facial Hydrating Spray™ or Essential Nine™ Eyelid & Skin Therapy ("E9").

LOOK AND FEEL REFRESHED!

These kits can also be used to create cold therapies for allergy and sinus pressure relief.

RECOMMENDED FOR

- Mild Dry Eye
- Allergies
- Jet lag
- Sinus pressure



D.E.R.M.™ Wellness Kits available with or without Comfort Cover™



D.E.R.M.™ with Gentle Formula Tea Tree Eyelid & Facial Cleanser™ and E9



D.E.R.M.™ with Anti-Aging Formula Tea Tree Eyelid & Facial Cleanser™ and E9



D.E.R.M.™ with Gentle Formula Tea Tree Eyelid & Facial Cleanser™ and Relax Formula Facial Hydrating Spray™



D.E.R.M.™ with Advanced Formula Tea Tree Eyelid & Facial Cleanser™ and Soothing Formula Facial Hydrating Spray™



Shown: D.E.R.M.™ with Comfort Cover™



MODERATE AND SEVERE OCULAR SURFACE DISEASE



CONTROLLED MOISTURE AND HEAT

Moderate to severe Dry Eye, MGD and blepharitis patients need up to 20 minutes of controlled moist-heat therapy for immediate and long-term relief. Our patented tranquileyes® and tranquileyes xl® systems are the only natural, scientifically proven home treatments on the market that deliver safe, controlled moist heat for up to 25 minutes.

The patented tranquileyes® and tranquileyes xl® systems incorporate two reusable gel pack technologies to create safe, controlled moist-heat therapies.

The combination of a heat source and water insulated in the thermal plastic goggle is the key to achieving the optimum temperature, humidity and duration required for immediate and long-term relief from symptoms of moderate to severe Dry Eye, MGD and blepharitis.

Anatomically designed for comfort, tranquileyes® and tranquileyes xl® eye covers are beautifully sculpted and made from medical grade, thermal plastic that is FDA approved.

We use viscoelastic foam - the highest quality foam available. This innovative type of memory foam is designed to seal and conform naturally, using the body's own heat to deliver a custom seal for each individual user.

Resting on the orbital bones (original) or over them (XL), the design allows for optimal blood flow to vital arteries above the nose without placing unnecessary pressure on the sensitive blood vessels under the eye.



PATENTED CONTROLLED MOIST-HEAT

This open-celled foam “breaths” and is latex-free, thereby minimizing the risk of skin irritation.

A uniquely designed comfort wrap holds the eye covers in place. The comfort wrap is wide in the back and narrows toward the temples for maximum comfort. The wrap is easily adjusted with two specially designed sliders, to ensure a custom fit and complete seal of the goggle to the eye area.



Creating effective moist-heat compresses for moderate to severe Dry Eye and blepharitis requires sustained temperature. The tranquileyes® and tranquileyes xl® systems incorporate heat sources to achieve the optimum temperature and duration required for immediate and long-term results.

Sustaining temperature requires that a heat source be introduced to the compress.

Eye Eco gel pack technologies have been scientifically proven to effectively improve tear quality by providing the intensity and duration of heat required to effectively melt the oils in the Meibomian glands, which allows for slower tear evaporation.

The increased humidity stabilizes the tear film, accelerates heat transfer to the lid margin, and hydrates the sensitive eyelid and surrounding skin, while slowing tear evaporation.



THE TRANQUILEYES® AND TRANQUILEYES XL® DIFFERENCE

THERMOEYES™ BEADS

Thermoeyes™ Beads are made of non-toxic glycerin and water. The Beads are preassembled in a fabric pocket which provides the moisture required to increase humidity while improving comfort. Once heated in the microwave, the pocketed Beads are placed into the back of the goggle, where they provide 12 to 25 minutes of controlled moist heat ranging between 102-110°F/39-43°C. Each set of Beads can be reused approximately 60 times for moist heat. They can also be used for cold therapies to reduce inflammation and puffiness.



THERMOEYES™ INSTANTS

Thermoeyes™ Instants are made of sodium acetate, a non-toxic, food grade salt water. Inside the Instants gel pack is a stainless steel disc. Heat is created by bending the stainless steel disc back and forth. The sound created by clicking the disc changes the molecular structure from liquid to crystal, thereby generating heat. Once activated, the gel pack is placed into a moistened fabric pocket then into the back of the goggle. The resulting controlled moist heat created ranges from 102-110°F/39-43°C and lasts 15 to 25 minutes. The gel packs can be reused over 150 times for soothing moist heat. It's easy to reuse thermoeyes™ Instants – just place in a continuous rolling boil for 3 to 5 minutes.



▶ Watch the Beads and Instants videos at www.eyeeeco.com

	BEADS	BEADS XL	INSTANTS	INSTANTS XL
MOIST-HEAT THERAPY DURATION	12-15 min	20-25 min	12-15 min	20-25 min
TEMPERATURE (°F)	102°-110°	102°-110°	102°-110°	102°-110°
RELATIVE HUMIDITY	94%-90%	92%-82%	94%-89%	94%-80%
NUMBER OF USES	60	60	150	150
RESETTING REQUIREMENTS	None	None	Boil 3 min	Boil 5 min
COLD THERAPY DURATION	10 min	15 min	10 min	12 min

● Shown: tranquileyes xl®
Lavender



MODERATE DRY EYE THERAPY



DOCTOR RECOMMENDED

Our patented tranquileyes® eye hydrating therapy is doctor-recommended and scientifically proven to relieve moderate dry eyes. Soothing 12 to 15 minute compresses sustain optimum humidity and temperature (between 102-110°F) that industry studies have shown provide immediate and long-term relief from moderate Dry Eye.

The masks can also be used to create cold therapies for eyelid puffiness, itchiness, and migraine headaches.

Interchanging thermoeyes™ Beads and thermoeyes™ Instants allows flexibility in choosing the treatment type that best works with your day.

Made in the USA.

RECOMMENDED FOR

Aqueous deficient Dry Eye
Sjögren's syndrome
Recurrent corneal erosions
Puffy eyelids
Jet lag

KIT OPTIONS





tranquileyes® Advanced with Beads & Instants



tranquileyes® with Beads



tranquileyes® with Instants

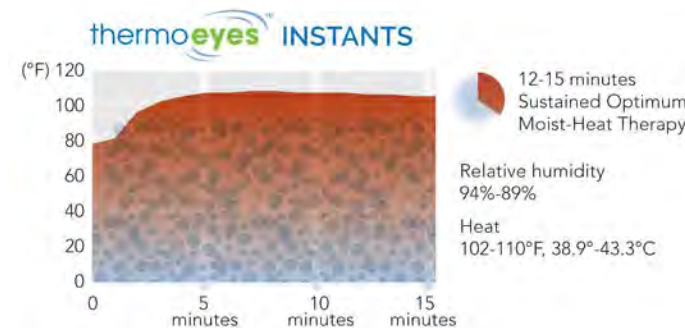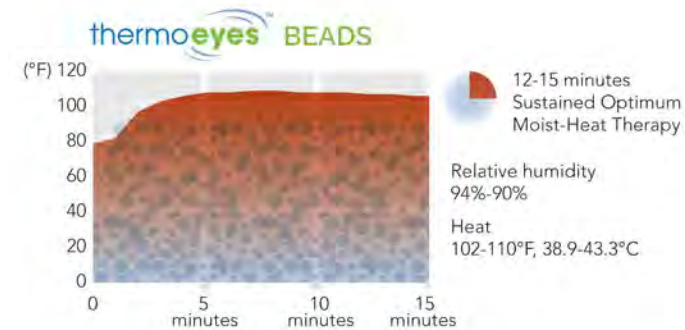

COLOR OPTIONS



ACCESSORIES



Spare Beads



Spare Instants



Spare Foam



Comfort Cover™



Spare Fabric Pockets

SEVERE DRY EYE THERAPY



PATENTED TECHNOLOGY

Our patented tranquileyes xl® delivers the most natural and effective moist-heat home treatment for severe Dry Eye on the market. The system combines the goggle, reusable gel packs and water to create 20-25 minutes of optimum temperature and humidity. Use tranquileyes xl® for immediate and long-term relief from severe Dry Eye, MGD or blepharitis.

Resting over the orbital bones, the XL moist heat therapy promotes circulation and speeds heat transfer, delivering controlled temperature between 102-110°F degrees.

The tranquileyes xl® is also used to create cold therapies for managing eyelid and under-eye puffiness, inflammation and ocular allergies. Made in the USA.

RECOMMENDED FOR

Moderate to severe evaporative Dry Eye (MGD)
Ocular allergies
Moderate to severe blepharitis
Demodex infestation
Lid margin inflammation
Eyelid and under-eye puffiness

KIT OPTIONS



tranquileyes xl® Advanced with Beads & Instants



tranquileyes xl® with Beads



tranquileyes xl® with Instants



COLOR OPTIONS



ACCESSORIES



Spare Beads XL



Spare Instants XL



Spare Foam



Comfort Cover™



Spare Pockets XL




TRANQUILEYES XL® CASE STUDIES

TRANQUILEYES XL® CASE STUDY 1

A 35-year-old female, working 8-10 hours in front of a computer screen in an air-conditioned office, complaining about Dry Eye discomfort was examined with the OCULUS Keratograph® 5M before and after using the tranquileyes xl® moist-heat therapy with Instants for 20 minutes.


NIK BUT RESULTS FOR THE RIGHT EYE	BEFORE TREATMENT	AFTER TREATMENT
Tearfilm Breakup (average)	7.48 seconds	22.94 seconds
Total time of NIKBUT Exam	10.11 seconds	24.92 seconds

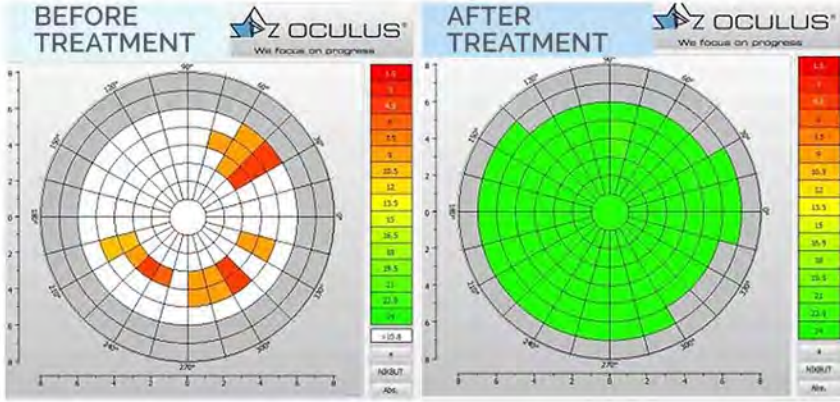

TRANQUILEYES XL® CASE STUDY 2

A 52-year-old female, working 9-11 hours in front of a computer screen in an air-conditioned office, complaining about Dry Eye discomfort was examined with the OCULUS Keratograph® 5M before and after using tranquileyes xl® moist-heat therapy with Instants for 15 minutes.

NIK BUT RESULTS FOR THE LEFT EYE	BEFORE TREATMENT	AFTER TREATMENT
Tearfilm Breakup (average)	8.32 seconds	14.83 seconds
Total time of NIKBUT Exam	11.02 seconds	24.13 seconds

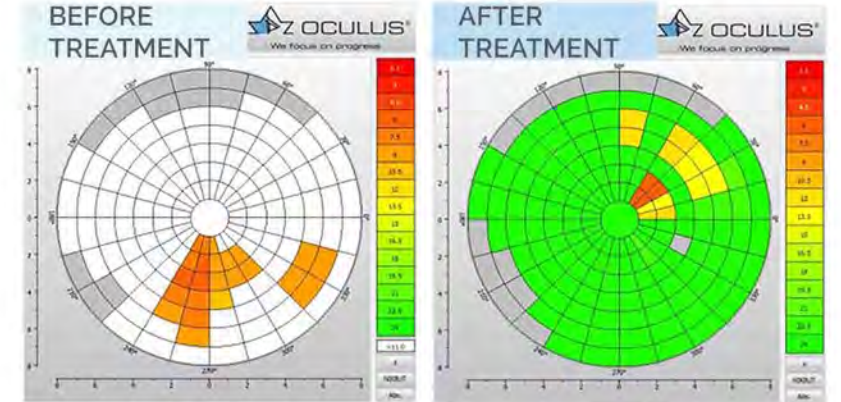

PATENTED MOIST-HEAT STYE KIT™



● Shown: Stye Kit™ Blue (Left Eye)

stye kit™

Perfect for the Dry Eye sufferer with a busy lifestyle, Stye Kit™ is a revolutionary hands-free method for naturally relieving stye and chalazion symptoms. A stye is an infection of an oil gland in the eyelid. A chalazion is a cyst in the eyelid due to a blocked oil gland. Both cause tenderness or pain, discomfort and a noticeable bump.

Our patented system delivers soothing 12 to 15 minutes of moist-heat therapy to one eye at a time. Secured by an elastic band, patients are free to continue with other activities such as reading, using the computer, or watching television, while applying their moist-heat compress. Made in the USA.

RECOMMENDED FOR

- Styes
- Chalazion
- Non-compliant Dry Eye

COLOR OPTIONS



Watch the stye kit™ video at www.eyeeeco.com



● Shown:
eyecLOUD®
with cotton
liner



EYECLOUD® PROFESSIONAL IN-OFFICE MOIST-HEAT



IN-OFFICE & POST-OP TREATMENT

Made from medical grade thermal plastic, our patent-pending eyecLOUD® is uniquely designed to comfortably deliver a 20 minute in-office treatment of controlled moist-heat to help loosen oils in the Meibomian glands.

The eyecLOUD® moist-heat treatment is recommended for use by doctors prior to performing a manual expression of the Meibomian glands, Lipiflow® or lid debridement. The eyecLOUD® also creates cold compresses for managing swelling and inflammation post-procedure, or post-operatively.

The system incorporates both the Instant and Beads reusable gel pack technologies for optimum performance. The gel packs are tucked into the back of the eyecLOUD® mask. A moistened disposable cotton liner is applied over the back of the gel packs to create moist-heat and for hygiene. The eyecLOUD® is then gently placed over the eyelids and secured with a soft elastic strap. The strap slips into a groove in the center of the mask, allowing for even distribution of pressure to the eyelids.

Each kit comes with the eyecLOUD® mask, four Instants gel packs, one Beads gel pack, one cotton sleeve, 200 liners and one 32 oz. hot pot (for use in resetting Instants).

The eyecLOUD® mask is made in the USA.

RECOMMENDED FOR

Before Lipiflow®
Before gland expression
Before lid debridement

COLOR OPTIONS



● Shown: eyeseals™ 4.0
Black



HYDRATING NIGHTTIME DRY EYE RELIEF

Are you a Dry Eye sufferer that also suffers from poor quality sleep and daytime tiredness? It's no coincidence. Poor quality of sleep is related to a disruption in the circadian rhythm, which has been reported to influence tear secretion and tear stability.¹

SLEEP APNEA

According to several studies, about 25 million adults in the U.S. have obstructive sleep apnea. Since it is estimated that only 10% of the population has been adequately screened for an appropriate diagnosis, the actual number of sleep apnea sufferers is likely much higher.

Compared to a normal population, sleep apnea patients have higher ocular irritation symptoms, abnormal tear break-up time and increased upper and lower lid laxity. There is also a correlation between normal-tension glaucoma and sleep apnea, due to the lack of oxygenation of the optic nerve head during sleep.

While the use of a CPAP device is widely considered the best sleep apnea treatment, one frequent complaint by CPAP users is that the mask releases air over the eyes, causing eye dryness, discomfort, and disruptions in sleep. Eye Eco products reduce or eliminate this unwanted side-effect to sleep apnea treatment providing a comfortable, uninterrupted sleep.

▶ Watch the nighttime Dry Eye relief video at www.eyeeeco.com

eyeseals™ 4.0

eyeseals™

eyeeeco™

LAGOPHTHALMOS

Nocturnal lagophthalmos is another condition associated with poor sleep and dry eyes. Impartial lid closure prevents the natural tears from spreading properly.² In addition, lagophthalmos patients are more vulnerable to recurrent corneal erosions.³ Some doctors believe the prevalence of lagophthalmos to have increased in the last few decades up to 10-20% of the population.⁴

FLOPPY EYE SYNDROME

Floppy eye syndrome patients have extensive lid laxity which has been linked to signs of Dry Eye due to tear film abnormality.⁵ Compared to the normal population, floppy eye syndrome patients may show decreased Schirmer's score, Meibomian gland drop out and symptoms of ocular pain similar to those of Dry Eye patients.⁶

¹Kawashima, Motoko, et al. "The association of sleep quality with dry eye disease: the Osaka study." *Clinical Ophthalmology* (Auckland, NZ) 10 (2016): 1015.

²Mastrota, Katherine. "Why So Dry?." *Optometric Management* 45.11 (2010): 33-34.

³Maharaj, Richard "Getting Closure: Exposing the Eyelid Overbite" *Eye on Eyes* (2012)

⁴Mastrota, Katherine M. "Diagnosing and treating lagophthalmos." (2015).

⁵Liu, Daniel Tzong-Shyue, et al. "Tear film dynamics in floppy eyelid syndrome." *Investigative ophthalmology & visual science* 46.4 (2005): 1188-1194.

⁶Chhadva, Priyanka, et al. "The Impact of Eyelid Laxity on Symptoms and Signs of Dry Eye Disease." *Cornea* 35.4 (2016): 531.

ON YIX™

tranquileyes®

QUARTZ™

PROTECTIVE EYESEALS™ AND EYESEALS™ 4.0

eyeseals™ eyeseals™ 4.0



Soft and flexible, eyeseals™ and eyeseals™ 4.0 are made from medical grade thermal plastic and proven to help relieve dry eyes during rest and sleep. The unitary design gently rests over the orbital bones, leaving room to blink while sheltering eyes from drafts, dust, low humidity environments and leaking air from CPAPs.

Fitted with an adjustable microfiber head wrap, the masks comfortably keep the dry air out while creating a moisture rich environment for the eyes, eyelid and surrounding skin.

The eyeseals™ feature a protective ridge above the nose to improve comfort and prevent chaffing from the CPAP mask.

The masks are hypoallergenic and latex-free. Made in the USA.

RECOMMENDED FOR

- Lagophthalmos (except clear)
- CPAP users
- Floppy eye syndrome
- Recurrent corneal erosions

COLOR OPTIONS

- eyeseals™ ● ○
- eyeseals™ 4.0 ● ● ● ● ● ● ● ● ○

IMPROVING DRY EYE COMFORT AT NIGHT

QUARTZ™ ONYX™

Lightweight and comfortable, the Onyx™ (black) and Quartz™ (clear) offer lagophthalmos and sleep apnea patients nighttime relief from associated Dry Eye. Soft and flexible, these hydrating masks are made from medical grade silicone and rest inside the orbital bones. Fitted with an adjustable head wrap made of microfiber, they gently protect the eyes from drafts and low humidity environments. They fog up during rest or sleep, creating a moisture-rich environment perfect for relieving Dry Eye and related conditions. Recommended for smaller faces and children.

The masks are hypoallergenic and latex-free. Made in the USA.

RECOMMENDED FOR

- Lagophthalmos (Onyx™)
- CPAP users
- Floppy eye syndrome
- Air travel

COLOR OPTIONS

- ○



● Shown: eyeseals™ 4.0 Clear



● Shown: Onyx™



● Shown: tranquileyes®
Black

TRANQUILEYES®: SLEEP COMFORTABLY - WAKE REFRESHED



The tranquileyes® goggles are designed to promote relaxing, restful sleep, making them perfect for long flights, power naps, and regular sleep patterns.

Padded with comfortable memory foam, tranquileyes® goggles naturally shelter your eyes from drafts, dust, and low humidity environments, all contributors to Dry Eye and related conditions.

RECOMMENDED FOR

Nighttime Dry Eye
Lagophthalmos
Recurrent corneal erosions
Air travel

COLOR OPTIONS



tranquileyes® Mini.
Contents: tranquileyes® goggle,
stainless steel travel canister.

The tranquileyes® goggle includes a set of removable foam inserts which can be soaked in hot water prior to sleep. This aids in the creation of moisture, a key to soothing tired eyes. For migraine headaches or puffiness, the pads can be frozen to create effective cold therapies, anytime you need them.

The goggle also creates total darkness, stimulating the pineal gland to release melatonin, a key promoter of deep sleep and relaxation.

Kits available with just the tranquileyes® goggle or with the goggle and accessories.



tranquileyes® Travel and Sleep.
Contents: tranquileyes® goggle, one set of foam replacements,
one mesh bag for cleaning.



ASSURING THE BEST FIT FOR REST OR SLEEP

tranquileyes®



tranquileyes® xl



QUARTZ™ ONYX™



eyeseals™



eyeseals™4.0



PROPRIETARY TEA TREE EYELID & FACIAL CLEANSERS

- Hydrating non-irritating tea tree cleansers for eyelids and face



TEA TREE EYELID & FACIAL CLEANSERS

TeaTree EYELID & FACIAL CLEANSER

THE EFFICACY OF TEA TREE

Demodex mites are tiny parasites that live in the Meibomian glands of the eyelids and the sebaceous glands of the skin, meaning they are not only found on eyelashes but also other face areas. *Demodex* can cause uncomfortable symptoms including eye irritation, eyelid inflammation, itching and loss of eyelashes. Eyelid hygiene is crucial for management of *Demodex*-caused conditions such as blepharitis. Numerous scientific studies have reported the inefficacy of known methods such as the use of baby shampoo in the treatment of conditions related to *Demodex*. Tea tree oil has been proven to be one of the only known products to inhibit *Demodex* growth.

HYDRATING AND NON-IRRITATING FORMULAS

Our proprietary Tea Tree Eyelid & Facial Cleansers are perfect for those with sensitive skin. Formulated to be non-irritating to the eyes, the tea tree concentrations cleanse, hydrate and soothe. Helping to manage *Demodex*, Tea Tree Eyelid & Facial Cleansers are doctor-recommended for use on the eyelids and face twice daily for optimum results.

Infused with naturally hydrating ingredients, chamomile and shea butter, these cleansers gently remove all dirt and make-up and leave the skin feeling smooth, silky and refreshed. Made in the USA and formulated without the use of parabens. All formulas are vegan and gluten-free.

ADVANCED FORMULA

Formulated with 2% tea tree oil, the Advanced Formula is recommended for moderate to severe cases of lid inflammation, rosacea and acne, or for those who desire a deeper cleanse.

RECOMMENDED FOR

Demodex maintenance

Deeper cleansing

Moderate to severe blepharitis

Moderate to severe rosacea and acne

Styes and chalazion



ADVANCED FORMULA, 2% TEA TREE

Available in
180 ml, ±370 applications and
50 ml, ±90 applications

TEA TREE EYELID & FACIAL CLEANSERS

GENTLE FORMULA

The Gentle Formula contains a mild concentration of 1% tea tree oil to help manage *Demodex* and reduce blemishes. Infused with chamomile and shea butter, it gently removes all dirt and make-up without eye irritation or drying the skin.

RECOMMENDED FOR

Demodex prevention

Daily lid maintenance

Rosacea, facial acne, blemishes

Lid and facial make-up removal



GENTLE FORMULA, 1% TEA TREE

Available in
180 ml, ±370 applications and
50 ml, ±90 applications

ANTI-AGING FORMULA

Our Anti-Aging Formula has the added benefits of Regu-Age®, a protein complex that visibly reduces dark circles and eye puffiness. Cleanse, rehydrate and refresh your eyes with our Tea Tree Eyelid & Facial Cleanser™ Anti-Aging Formula.

RECOMMENDED FOR

Demodex prevention

Daily lid maintenance

Rosacea, facial acne, blemishes

Lid and facial make-up removal

Reducing the appearance of dark circles and puffiness



ANTI-AGING FORMULA, 1% TEA TREE

Available in
180 ml, ±370 applications and
50 ml, ±90 applications

ESSENTIAL NINE™ EYELID & SKIN THERAPY

E9 EYELID & SKIN THERAPY

Essential Nine™ Eyelid & Skin Therapy is formulated to be applied directly on the eyelids and sensitive surrounding skin. The eyelid, the thinnest skin on the body, is designed to protect and hydrate the ocular surface. The eyelid can dry easily causing puffiness and discomfort to the eyes.

Essential Nine™ includes aloe, chamomile, green tea and clinically proven active ingredients to deeply hydrate and reduce the visible signs of aging around the eye. E9 is unscented, non-irritating and absorbs quickly, leaving the skin feeling silky and smooth.



Essential Nine™
Eyelid and Skin Therapy

Available in
1.1 oz / 30 ml

RECOMMENDED FOR

Dark circles
Fine lines
Eyelid puffiness
Eyelid hydration

ESSENTIAL NINE™ ACTIVE INGREDIENTS

EYESERYL® peptide has the ability to correct fluid retention in the facial areas and dramatically reduce the decomposition of collagen. The result: eye puffiness is reduced and skin texture is improved.

SODIUM HYALURONATE not only hydrates the skin, but helps to reduce water loss, giving the skin a young fullness that helps to minimize the appearance of fine lines and wrinkles.

REGU-AGE® is a clinically proven formula that visibly reduces the appearance of dark circles.

GATULINE® A is a soothing ingredient which boosts microcirculation around the eyes and helps reduce redness and inflammation.

PENTAVITIN® is a carbohydrate complex that ensures instant and long-lasting deep hydration.

FACIAL HYDRATING SPRAYS



Facial Hydrating™ SPRAY

Enhance the benefits of our moist-heat and cold therapies with one of four specially formulated Facial Hydrating Sprays. Simply mist over the face or over foam inserts prior to wearing the tranquileyes® goggle.

Each formula is all natural and offers a

specific benefit. All include active ingredient aloe (to intensely hydrate skin) and allantoin (to promote skin healing). All are formulated for maximum comfort and non-irritating to the eyes.

Available in 2 oz / 59 ml.

RECOMMENDED FOR

Puffy eyelids (Puffy Eye Formula)
Under eye puffiness (Puffy Eye Formula)
Fine lines (Age Defense formula)
Eyelid hydration
Jet lag

SOOTHING EYE MIST

Specially formulated for use around eyes, our Soothing Eye Mist is unscented, non-irritating, and hydrating for the eyelid. Use as a substitute for water in Eye Eco therapies.



AGE DEFENSE FORMULA

Combines the antioxidants of green tea, vitamin C and honeysuckle to help protect skin.



RELAX FORMULA

Infused with soothing lavender essential oil to calm your mind and ease stress.



PUFFY EYE FORMULA

Infused with the moisturizing and anti-inflammatory benefits of cucumber and aloe to calm the skin and reduce puffiness.





● Shown: Small Moisture Release Eyewear™ Red Tortoise Frame with Copper Lenses

MOISTURE RELEASE EYEWEAR™ FOR DAYTIME AND SUN

moisture release
E Y E W E A R™

Our comfortable and stylish Moisture Release Eyewear™ provides relief from dryness and ocular allergies indoors and outdoors, so you can stay active and comfortable.

Developed and patented by a Board Certified ophthalmologist, they shield eyes from the environment, create a seal against dust and pollen, and add moisture to keep eyes hydrated and happy.

The stylish, lightweight frames utilize two removable cartridges that can be soaked in water and placed into the side of the frame. Body heat causes the water in the cartridges to evaporate, increasing the relative humidity around the eyes. A comfortable thermal plastic gasket and bendable nose bridge seal the area, which helps slow the evaporation of tears for up to 4 hours at a time. An anti-fog coating in the polycarbonated lenses prevents the glasses from fogging-up.

The Moisture Release Eyewear™ system soothes dry eyes, improves comfort, reduces fatigue and improves visual acuity so you don't miss a thing. Just slide them on and keep going!

The eyewear is available in two sizes and includes two sets of cartridges for 240+ uses. Both sizes are Rx ready and support prescriptions of ±3.0 diopters (large frame) and ±4.0 diopters (small frame). You can also choose clear lenses or sunglasses, which provide 100% UVA and UVB protection.

RECOMMENDED FOR

- Aqueous deficient Dry Eye
- Sjögren's syndrome
- Ocular allergies



Adjustable temples

Watch the MRE video at www.eyeeeco.com



Large frame



Small frame

FRAME COLOR OPTIONS



LENSES COLOR OPTIONS



	SMALL FRAME	LARGE FRAME
RX RANGE	±4.0	±3.0
EYE SIZE	57 mm	60 mm
TEMPLE LENGTH	125-145 mm	120 mm
UVA AND UVB PROTECTION	100%	100%

ACCESSORIES



Cartridge, Demi Amber



Cartridge, Matte Black



Cartridge, Red Tortoise



Cartridge, Shiny Black



Cartridge, Yellow Tortoise



DOCTOR RECOMMENDED DRY EYE HOME TREATMENTS

MILD DRY EYE AND PREVENTION

- | | |
|--|---------------------------------|
| 1. Gentle or Anti-Aging Tea Tree Eyelid & Facial Cleanser™ | 1. Clean eyelids and face daily |
| 2. D.E.R.M.™ | 2. 5-9 minutes moist-heat daily |

MODERATE MGD, AQUEOUS DEFICIENT DRY EYE AND SJÖGREN'S SYNDROME

- | | |
|---|---|
| 1. Advanced 2% Tea Tree Eyelid & Facial Cleanser™ | 1. Clean eyelids and face daily |
| 2. tranquileyes® with Beads or Instants | 2. 15 minutes moist-heat daily |
| 3. Essential Nine™ Eyelid & Skin Therapy | 3. Apply to eyelid and surrounding skin daily |
| 4. Moisture Release Eyewear™ | 4. Daytime shelter |

SEVERE MGD

- | | |
|---|---|
| 1. Advanced 2% Tea Tree Eyelid & Facial Cleanser™ | 1. Clean eyelids and face daily |
| 2. tranquileyes xl® with Beads or Instants | 2. 10-20 minutes moist-heat daily |
| 3. Essential Nine™ Eyelid & Skin Therapy | 3. Apply to eyelid and surrounding skin daily |

STYES/CHALAZION

- | | |
|---|--|
| 1. Advanced 2% Tea Tree Eyelid & Facial Cleanser™ | 1. Clean eyelids and face daily |
| 2. Styte Kit™ with Instants | 2. 10-15 minutes moist-heat up to 4 times daily or as directed by doctor |

OCULAR ALLERGIES

- | | |
|--------------------------------|--|
| 1. tranquileyes xl® with Beads | 1. 10-15 minutes cold therapy once daily as needed |
| 2. Moisture Release Eyewear™ | 2. Daytime shelter |

LAGOPHTHALMOS

- | | |
|--|--------------------------------------|
| 1. Gentle Tea Tree Eyelid & Facial Cleanser™ | 1. Clean eyelids and face daily |
| 2. Onyx™ Silicone Shield, eyeseals™ 4.0 black or tranquileyes® | 2. Use nightly for hydrating shelter |

SLEEP APNEA (CPAP LEAKS)

- | | |
|--|--------------------------------------|
| 1. Gentle Tea Tree Eyelid & Facial Cleanser™ | 1. Clean eyelids and face daily |
| 2. Quartz™ Silicone Shield, or eyeseals™ clear, or eyeseals™ 4.0 clear | 2. Use nightly for hydrating shelter |

RECURRENT CORNEAL EROSIONS

- | | |
|--|--------------------------------------|
| 1. Gentle Tea Tree Eyelid & Facial Cleanser™ | 1. Clean eyelids and face daily |
| 2. tranquileyes® with Beads or Instants | 2. 10 minutes moist-heat daily |
| 3. tranquileyes® or eyeseals™ 4.0 | 3. Use nightly for hydrating shelter |

GLAUCOMA/SKIN DISCOLORATION

- | | |
|--|---|
| 1. Gentle Tea Tree Eyelid & Facial Cleanser™ | 1. Clean eyelids and face daily |
| 2. Essential Nine™ Eyelid & Skin Therapy | 2. Apply to eyelid and surrounding skin daily |

MODERATE TO SEVERE BLEPHARITIS

- | | |
|---|---|
| 1. Advanced 2% Tea Tree Eyelid & Facial Cleanser™ | 1. Clean eyelids and face daily |
| 2. tranquileyes xl® with Beads or Instants | 2. 10-20 minutes moist-heat daily |
| 3. Essential Nine™ Eyelid & Skin Therapy | 3. 10 minutes cold therapy daily |
| | 4. Apply to eyelid and surrounding skin daily |

DOCTOR RECOMMENDED DRY EYE HOME TREATMENTS

DEMODEX INFESTATION

- | | |
|---|-----------------------------------|
| 1. Advanced 2% Tea Tree Eyelid & Facial Cleanser™ | 1. Clean eyelids and face daily |
| 2. tranquileyes xl® with Beads or Instants | 2. 10-15 minutes moist-heat daily |

LID MARGIN INFLAMMATION

- | | |
|--|---|
| 1. tranquileyes xl® with Beads | 1. 10-15 minutes cold therapy as needed |
| 2. Essential Nine™ Eyelid & Skin Therapy | 2. Apply to eyelid and surrounding skin daily |

EYELID HYDRATION

- | | |
|--|---|
| 1. Essential Nine™ Eyelid & Skin Therapy | 1. Apply to eyelid and surrounding skin daily |
| 2. Soothing Eye Mist, Age Defense, Puffy Eye and Relax Facial Hydrating Spray™ | 2. As needed throughout the day |

PUFFY EYELIDS

- | | |
|--|---|
| 1. tranquileyes® with Beads | 1. 10 minutes cold therapy |
| 2. Puffy Eye Facial Hydrating Spray™ | 2. Inflammation treatment |
| 3. Essential Nine™ Eyelid & Skin Therapy | 3. Apply to eyelid and surrounding skin daily |

EYELID AND UNDER EYE PUFFINESS

- | | |
|--|---|
| 1. tranquileyes xl® with Beads | 1. 15 minutes cold therapy |
| 2. Puffy Eye Facial Hydrating Spray™ | 2. Inflammation treatment |
| 3. Essential Nine™ Eyelid & Skin Therapy | 3. Apply to eyelid and surrounding skin daily |

FINE LINES

- | | |
|--|---|
| 1. Anti-Aging Tea Tree Eyelid & Facial Cleanser™ | 1. Clean eyelids and face daily |
| 2. tranquileyes xl® with Beads or Instants | 2. 5-20 minutes moist-heat therapy |
| 3. Age Defense Facial Hydrating Spray™ and Essential Nine™ Eyelid & Skin Therapy | 3. Apply to eyelid and surrounding skin daily |

DARK CIRCLES

- | | |
|--|--|
| 1. Anti-Aging Tea Tree Eyelid & Facial Cleanser™ | 1. Clean eyelids and face in the morning and night |
| 2. tranquileyes xl® with Beads or Instants | 2. 10 minutes cold therapy in the morning and 10 minutes moist-heat at night |
| 3. Essential Nine™ Eyelid & Skin Therapy | 3. Apply to eyelid and surrounding skin daily |

LASH EXTENSIONS

- | | |
|--|--------------------------------------|
| 1. Gentle Tea Tree Eyelid & Facial Cleanser™ | 1. Clean eyelids and face daily |
| 2. eyeseals™ 4.0 black | 2. Use nightly for hydrating shelter |

JET LAG

- | | |
|--|----------------------------------|
| 1. tranquileyes® with Beads or Instants | 1. 10 minutes moist-heat therapy |
| 2. Soothing Eye Mist, Age Defense, Puffy Eye and Relax Facial Hydrating Spray™ | 2. Facial hydration |

DRY EYE RELIEF DROP-SHIP PROGRAM

We at Eye Eco are passionate about helping patients with mild, moderate and severe Dry Eye, feel better, naturally. Our drop-ship program provides doctors treating Dry Eye, complete access to our full line of scientifically proven products and therapies.

This program makes it easier than ever to help patients comfortably manage their Dry Eye symptoms at home.

HOW IT WORKS:



THE DOCTOR

- ✓ DIAGNOSES TYPE AND CAUSE OF DRY EYE
- ✓ PRESCRIBES EYE ECO HOME TREATMENTS FOR PATIENT
- ✓ ORDERS PRODUCTS FROM EYE ECO FOR DELIVERY TO PATIENT



EYE ECO

- ✓ PROCESSES ORDER AND BILLS PRACTICE
- ✓ THEN DROP-SHIPS PRODUCT TO PATIENT
- ✓ FOLLOWS UP WITH PATIENT 24-48 HOURS AFTER DELIVERY



PATIENT

- ✓ RECEIVES PERSONALIZED CARE WHILE IN THE COMFORT OF THEIR OWN HOME
- ✓ GAINS IMMEDIATE AND LONG-TERM RELIEF FROM THEIR SYMPTOMS
- ✓ ASSURED THEIR PRODUCTS ARE BACKED BY A 100% SATISFACTION GUARANTEED

EXCEPTIONAL CUSTOMER SERVICE FOR DOCTORS AND PATIENTS

WELLNESS CALL

Eye Eco now provides a complimentary Wellness Call for patients whose doctors have prescribed our products. An Eye Eco expert is available to contact patients within 24-48 hours of receiving their product to answer any questions.

Participation is easy. If you have a patient you would like Eye Eco to contact, please send an email to customerservice1@eyeeco.com. Include patient name, products prescribed, and their preference for being contacted (phone or email). We will acknowledge receipt of your request and let you know when we have reached your patient. Thank you for treating Dry Eye patients!





05/15/2018